ClearSounds®  
Hear how good life can be

# Operating Manual ClearSounds® CR200 ClearRing Amplified Telephone Ring Signaler

# **TABLE OF CONTENTS**

## **PRIOR TO USE**

Pg. 3 ..... Congratulations

Pg. 3-4 ..... Important Safety Instructions

Pg. 5 ..... Location of Controls

## **INSTALLATION**

Pg. 6

## **SETTINGS**

Pg. 7

## **TECHNICAL SPECIFICATIONS**

Pg. 8

## **REGULATORY COMPLIANCE**

Pg. 8-9 ..... Important FCC Information for Customers

## **INDUSTRY CANADA**

Pg. 10-11

## **WARRANTY POLICY**

Pg. 11-12

## PRIOR TO USE

Congratulations on your purchase of the CR200 Telephone Ring Signaler from ClearSounds Communications®. Please read this User Manual carefully in order to get the best use out of your new product. Keep this manual near the telephone for easy reference.

Please visit our website at [www.clearsounds.com](http://www.clearsounds.com) to register your product, download additional manuals and review troubleshooting tips.

Be sure to save your sales receipt as your proof of purchase should you need warranty service.

Warranty Service – This product is designed to provide years of quality service. However, if there is a malfunction, follow the warranty procedure on pages.

## IMPORTANT SAFETY INSTRUCTIONS

Read and understand the Installation Guide and User Manual. When using your telephone equipment, basic safety precautions should always be followed to reduce the risk of fire, electric shock and injury to person, including the following:

**WARNING:** The CR200 Ring Signaler is capable of amplifying sounds to a loud volume. It is important to instruct all potential users of its proper operation.

**WARNING:** Use only the power supply indicated in the User's Manual.

**WARNING:** To avoid the possibility of electrical shock, do not use this product near water (For example, near a bathtub, wash basin, kitchen sink or laundry tub, in a wet basement or near a swimming pool) while in the bathtub or shower or when you are wet. If the phone becomes submerged in water, do not

attempt to retrieve it until after you have unplugged the line cord from the modular wall jack. Do not plug the phone back in until it has dried completely.

**WARNING:** Avoid using the telephone during electrical storms. There may be a remote risk of electric shock from lightning. If the electrical storm is present in your immediate area, possibility of electrical shock exists.

**WARNING:** Do not use the phone if you suspect a gas leak in the area. The phone's electrical contact could create a spark, which could ignite any heavy concentration of leaking gas.

**WARNING:** When cleaning, unplug the ring signaler, and then use a damp cloth to wipe. Do not use liquid cleaners or aerosol cleaners.

**WARNING:** No telephone should ever be placed near an open radiator or heating register.

**WARNING:** Occasionally a power surge can occur on electrical lines without warning. With any electrical device that you plug into an outlet, it is suggested you use a surge protector. A surge protector will prevent power surges from damaging your phone. Surge protectors are simple devices available at your local hardware store.

**WARNING:** Do not overload wall outlets or extension cords as this can increase the risk of fire or electrical shock.

**WARNING:** Never push any object through the slots in the telephone. They can touch dangerous voltage points or short out parts that could result in risk of fire or electrical shock.


**WARNING:** Do not cover slots and opening on the units, they are needed for ventilation and protection against overheating.

**WARNING:** Ring volume is very loud and can damage your hearing. Never put your ear close to the ringer.

**IMPORTANT:** YOUR RECEIPT IS PART OF YOUR WARRANTY AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.


# PLEASE SAVE THESE INSTRUCTIONS

## LOCATION OF CONTROLS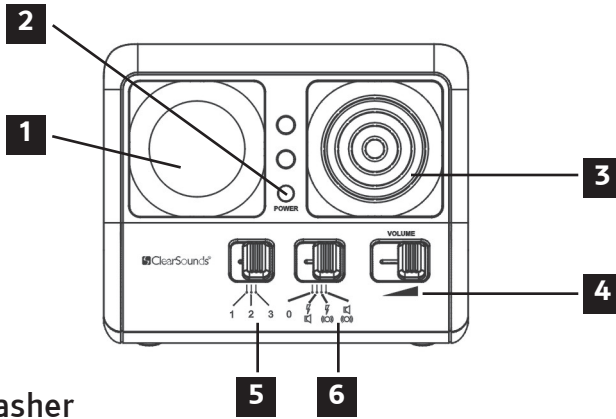

01. LED Flasher
02. Power Indicator
03. Speaker (Audible Ringer)
04. Ringer Volume Control
05. Ringer Mode Switch
06. Ringer Tone Switch

## BACK SIDE

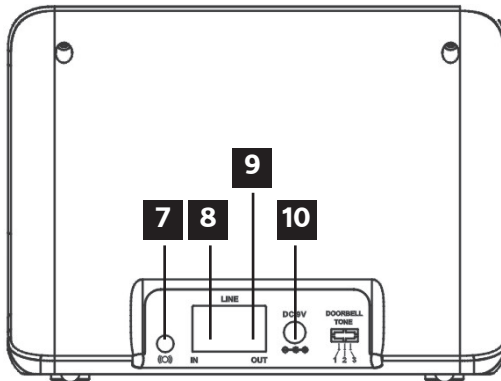07. Jack for (optional) ClearSounds Bed Shaker
08. Jack for Telephone Connection
09. Jack for Telephone Line Connection
10. AC Power Adapter Jack

# INSTALLATION


Installation of your **ClearSounds® CR200 ClearRing Amplified Telephone Ring Signaler** is easy.

If at any time you have questions about your ring signaler after reading this manual, please contact our ClearSounds® Customer Service Department at 1-800-965-9043.

## PARTS CHECKLIST

Check the contents of the box - The CR200 should contain:

1. Ring Signaler
2. AC Power Adapter
3. Telephone line cord
4. User's Manual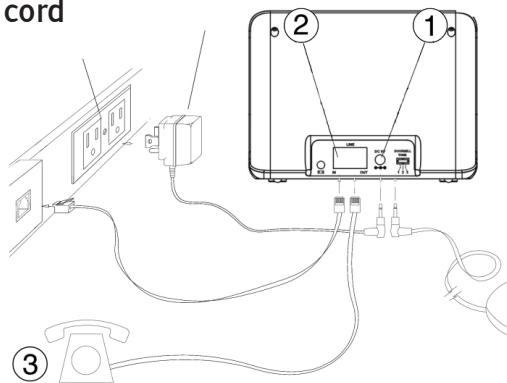


1. Connect the modular end of the AC power adapter into the 9V DC jack on the back of the CR200 and then plug the other end into a standard AC wall outlet.
2. Connect one end of the telephone cord to the LINE IN jack on the back of the CR200 and connect the other end to a modular phone jack.
3. Optional: Connect your telephone to the LINE OUT jack on the back of the CR200. **Note: The CR200 can operate independently without a telephone.**
4. Optional: Connect the optional bed shaker into the shaker jack on the back of the CR200 and place the shaker under your pillow or mattress.

Your CR200 is ready for use. Next, simply adjust the settings to fit your needs.

# SETTINGS

## Set the Ring Notification Mode

You can adjust the type of notification that is desired. Choose from LED Flashing, Audible Ringing, and Vibrating Bed Shaker (shaker optional) **Adjust the switch to the desired notification method(s).**

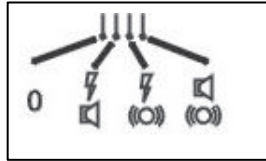The ring notification feature is turned off

The unit flashes and rings

The unit flashes and vibrates the bed shaker (when the shaker is connected)

The unit rings and vibrates the bed shaker (when the shaker is connected)


## Set the Ringer Volume Control

You can adjust the ringer volume as high as 95 dB. To adjust the volume, slide the ringer volume control to the right to make it louder and to the left to make it softer. 

## Set the Ringer Tone Control

The tone control shapes the sound by changing the frequencies of the ringer.

Select from the 3 available tone options.

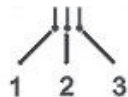# TECHNICAL SPECIFICATIONS

General Number: CR200  
Amplification: Up to 95 dB

## Power Requirements

### AC power Adapter:

Input: 100~240V 50/60Hz 450mA  
Output: 9.0VDC 1000mA

*Features and Specifications subject to change without notice.*

To learn more, call ClearSounds at 1-800-965-9043 or  
visit [www.clearsounds.com](http://www.clearsounds.com)

# FCC WANTS YOU TO KNOW

1. This equipment complies with Part 68 of the FCC rules and the requirements adopted by the ACTA. On the bottom of this equipment is a label that contains, among other information, a product identifier. If requested, this number must be provided to the telephone company.
2. If the **ClearSounds® CR200 ClearRing Amplified Telephone Ring Signaler** causes harm to the telephone network, the telephone company will notify you in advance that temporary discontinuance of service may be required. But if advance notice isn't practical, the telephone company will notify the customer as soon as possible. Also, you will be advised of your right to file a complaint with the FCC if you believe it is necessary.
3. The telephone company may make changes in its facilities, equipment, operations or procedures that could affect the operation of the equipment. If this happens, the telephone company will provide advance notice in order for you to make necessary modification to maintain uninterrupted service.

4. If you experience trouble with this equipment, you should disconnect it from the network until the problem has been corrected or until you are sure that the equipment is not malfunctioning.
5. Please follow instructions for repairing if any (e.g. battery replacement section); otherwise do not alternate or repair any parts of device except as specified.
6. Connection to party line service is subject to state tariffs. Contact the state public utility commission, public service commission or corporation commission for information.
7. If the telephone company requests information on what equipment is connected to their lines, inform them of:
  - a) The telephone number that this unit is connected to,
  - b) The ringer equivalence number
  - c) The USOC jack required [RJ11C], and
  - d) The FCC Registration Number

Items (b) and (d) are indicated on the label. The ringer equivalence number (REN) is used to determine how many devices can be connected to your telephone line. In most areas, the sum of the RENs of all devices on any one line should not exceed five (5.0). If too many devices are attached, they may not ring properly.

### Service Requirements

In the event of equipment malfunction, all repairs should be performed by our Company or an authorized agent. It is the responsibility of users requiring service to report the need for service to our Company or to one of our authorized agents. Service can be facilitated through our office at:

ClearSounds Communications, Inc.  
1743 Quincy Avenue, Suite 155 Naperville, IL 60540 U.S.A

## INDUSTRY CANADA CS03 STATEMENT

This product meets the applicable Industry Canada technical specification. This is confirmed by the registration number. The abbreviation, IC before the registration number signifies that registration was performed based on a Declaration of Conformity indicating that Industry Canada technical specifications were met. It does not imply that Industry Canada approved the equipment.

Before installing this equipment, users should ensure that it is permissible to be connected to the facilities of the local telecommunications company. The equipment must also be installed using an acceptable method of connections. The customer should be aware that compliance with the above conditions may not prevent degradation of service in some situations.

Repairs to certified equipment should be made by an authorized Canadian maintenance facility designated by the supplier. Any repairs or alterations made by the user of this equipment, or equipment malfunctions, may give the telecommunication company cause to request the user to disconnect the equipment.

**WARNING!** Users should not attempt to make such connections themselves, but should contact the appropriate electric inspection authority, or an electrician, as appropriate.

**CAUTION:** The Ringer Equivalence Number (REN) assigned to each terminal device provides an indication of the maximum number of terminals allowed to be connected to a telephone interface.

The termination on an interface may consist of any combination of devices subject only to the requirement that the sum of the Ringer Equivalence Numbers of all the devices does not exceed

5. An alternative indication of the REN is the Load Number (LN); this number should be divided by 20 to approximate the equivalent REN. The REN Number of this unit is indicated on the bottom of your phone.

If your home has specially wired alarm equipment connected to the telephone line, ensure the installation of the CR200 does not disable alarm equipment. If you have questions about what will disable alarm equipment, consult your telephone provider or a qualified installer.

This Class B digital apparatus complies with Canadian ICES-003.

## **WARRANTY POLICY**

### **LIMITED ONE-YEAR WARRANTY**

The product is warranted by ClearSounds against manufacturing defect in material and workmanship under normal use for one (1) year from the date of purchase. Should you experience a problem, contact our customer service department or visit our website at [www.clearsounds.com](http://www.clearsounds.com).

Be sure to save your sale receipt as proof of purchase date should you need warranty service.

Your CR200 Ring Signaler comes with a (1) year limited warranty from the date of purchase. This product is warranted by ClearSounds against manufacturing defect in material and workmanship under normal use. In the event that this product fails to function properly within one year of the original purchase due to defects in materials or workmanship, return the unit (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to ClearSounds Communications. ClearSounds Communications will either repair or replace the unit (with a refurbished unit or unit of equal condition) and return it you (using UPS/USPS ground shipping) at no cost to you. The warranty replacement or repair will be warranted for a period of 90 days or the remainder of the original warranty period (from original purchase date); whichever is longer.

The warranty does not cover accidents, negligence or breakage to any parts. This includes shipping damage, failure to follow instructions, misuse, fire, floods, use of incompatible accessories, Acts of God or failure in your phone service carrier's line service. The product must not be tampered with or taken apart by anyone who is not an authorized ClearSounds Communications® representative. Tampering with the phone will void any written or implied warranties.

If a defect covered by this warranty should occur, promptly contact a Customer Service Representative by phone or log onto [www.clearsounds.com](http://www.clearsounds.com) to obtain a Return Merchandise Authorization number and shipping instructions before shipping the product to us. Any shipment without a (RMA) Return Merchandise Authorization number will not be accepted and will be returned to you at your expense. All authorized returned products must be accompanied with proof of purchase and a brief explanation of the problem.

For out of warranty repairs and service, please contact our Customer Service Department for directions. The repair shall be warranted for 90 days.

**IMPORTANT: YOUR RECEIPT IS PART OF YOUR WARRANTY AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.**

**Service in United States and Canada, contact:**

ClearSounds Communications  
Attention: Repair Center  
1743 Quincy Avenue, Suite 155  
Naperville, IL 60540  
USA  
800-965-9043

Check our website [www.clearsounds.com](http://www.clearsounds.com) for additional information and to download additional manuals and information.

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