
# WakeAssure III

#### Snooze / Flashlight -Low Batteru AM Zz • iO: ALARM ON PM icon Timer/ **STOPWATCH** AL AM PM Stopwatch ON icon <sup>™</sup>Light Alarm TIMER ON **II** ■ Bed Shaker icon **☆ (O) •()** DOWN TIMER (ALSET) (ALSET) UNLOCK **UP/Test** SET AL SET/ TIMER<sup>T</sup> **SELECT** START / STOP (timer)

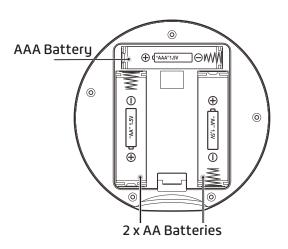

#### BATTERY INSTALLATION

Your Jolt requires 1 x AAA alkaline battery and 2 x AA alkaline batteries.

- 1. On the back of your unit, press the latch and remove battery door.
- 2. Install batteries as shown, ensuring polarity matches the diagram.
- 3. Close battery door; make sure latch clicks in place.

#### **IMPORTANT!**

**Auto Key Lock** feature is active after 1 minute of no buttons being pressed on the unit. This ensures that buttons cannot be pressed by mistake while traveling and/or while sleeping.

# TO UNLOCK THE CLOCK:

Press and hold the **DOWN/UNLOCK** button until you hear the unlock beep tones.

### SET TIME/SET SNOOZE DURATION

- Press and hold SET/TIMER until the hour digits start flashing; release SET/TIMER button. Press UP/DOWN to set the time in hours.
- 2. Press **SET/TIMER** again; you will see the minutes digits flashing. Press **UP/DOWN** to set the time in minutes.
- Press SET/TIMER again; press UP/DOWN to choose between 12 hour or 24 hour display for your Jolt (default setting is 24 hour display). If 12 hour is selected, press UP/DOWN to choose between AM and PM.
- 4. Press **SET/TIMER** again; now you will have the opportunity to choose the **SNOOZE** duration (default is 5 minutes). You can choose any timing between 5 minutes and 60 minutes.
- 5. Press **SET/TIMER** again to exit the time/snooze setup mode.

# SET ALARM (default is 6:00 AM)

- Press and hold AL/SET button until AL is displayed and the hour digits start flashing; release the AL/SET button. Press UP/DOWN to set the hour for your alarm time.
- 2. Press **AL/SET** again; you will see the minutes digits flashing. Press **UP/DOWN** to set the minutes for the alarm time.
- 3. Press **AL/SET** again; press **UP/DOWN** to choose the vibration level (HI/LO). The corresponding icons are displayed on the screen.

- 4. Press **AL/SET** again; press **UP/DOWN** tp choose the sound level (HI/LO). The corresponding icons are displayed on the screen.
- 5. Press **AL/SET** again to exit the alarm setup mode.

#### **SELECT ALARM MODE**

In idle mode, press **AL/SET** to toggle between the available alarm modes - see below.

# REMINDER - TO UNLOCK THE CLOCK: Press and Hold DOWN/UNLOCK button until it beeps

<b>4</b> )))	Audible alarm mode	
191	Bed shaker alarm mode	
÷Ö:	Light alarm mode	
	Audible + Bed shaker mode	
÷Ö: ◆»)	Audible + Light alarm mode	

#### SNOOZE/TURN OFF ALARM

- 6. To snooze: while alarm is active, press **SNOOZE** button to engage the snooze function; alarm will stop and come back on after snooze time expires.
- 7. To turn off: while alarm is active, press **AL** SEL/SELECT once; alarm will stop and be active again the following day at the same time.

#### **FLASHLIGHT**

Press and hold **SNOOZE/FLASHLIGHT** to turn on the Flashlight function. Press and hold **SNOOZE/FLASHLIGHT** again to turn it off.

## **TIMER FEATURE**

- 1. In idle mode, press **SET/TIMER** once to activate the timer/stopwatch feature. The timer icon is displayed on the screen, and the digits start flashing.
- 2. Press **DOWN** to set timer in hours, press **UP** to set timer in minutes.
- 3. Press **START/STOP** to start timer; you will see the ":" and **DOWN** icon flashing.

- 4. During timer countdown, press **START/STOP** to pause the timer; from here, press **START/STOP** to resume, OR press and hold **START/STOP** again to stop or reset the timer.
- 5. When the timer reaches 0:00, the alarm goes off. It then starts counting the time in order to display the elapsed time since timer expiration. Press **START/STOP** to stop the alarm.
- 6. **NOTE:** during timer countdown, press **SET/TIMER** to return to clock display. The timer icon flashes to indicate the active timer/stopwatch. To stop the timer in this mode, press and hold the **DOWN/UNLOCK** button.

#### STOPWATCH FEATURE

- In idle mode, press SET/TIMER once to activate the timer/stopwatch feature. The timer icon is displayed on the screen, and the digits start flashing. Press and hold START/STOP to reset display to "0:00".
- 2. Press **START/STOP** to start the stopwatch; you will see the ":" and **UP** icon flashing.
- 3. During stopwatch mode, press **START/STOP** to pause the stopwatch; from here, press **START/STOP** to resume, OR press and hold **START/STOP** again to stop or reset the stopwatch.
- 4. **NOTE:** during stopwatch mode, press **SET/TIMER** to return to clock display. The timer icon flashes to indicate the active timer/stopwatch.

### **LOW BATTERY**

When low battery icon is displayed, replace batteries with equivalent/alkaline only batteries. **NOTE:** frequent use of bed shaker feature with your alarm will shorten the life of the batteries.

#### **TROUBLESHOOTING**

If your Jolt does not function properly, reset the unit by removing/reinstalling the batteries. You will need to set the time again.

## **TEST MODE**

In order to quickly check the features of the unit, unlock the Jolt first (press and hold the **DOWN/UNLOCK** button to unlock), then press and hold the **TEST** button. The following features turn on for 5 seconds: bed shaker, light flashing, audible alarm.

#### SPECIFICATIONS

Alarm Duration	1 minute		
Snooze Duration	5 to 60 minutes	ninutes	
	Time format	24 hour	
	Time	0:00	
Default Settings	Alarm	6:00 (6:00 AM)	
Derault Settlings	Shaker/alarm level	Hi	
	Snooze duration	5 minutes	
	Timer/Stopwatch	23 hrs 59 mins	
Audible Alarm	80dBSPL at 1m		

#### WARRANTY

The following warranty and service information applies only to products purchased and used in the U.S. and Canada. For warranty information in other countries, please contact your local retailer or distributor.

Limited Warranty: Clarity Products LLC ("Clarity") warrants to the original consumer purchaser that, except for limitations and exclusions set forth below, this product shall be free from defects in materials and workmanship for a period of one (1) year from the date of original purchase ("Warranty Period"). The obligation of Clarity under this warranty shall be at Clarity's option, without charge, of any part or unit that proves to be defective in material or workmanship during the Warranty Period.

Please use the original container, or pack the unit(s) in a sturdy carton with sufficient packing material to prevent damage. Include the following information:

#### **Proof-of-purchase** indicating:

- · Model number and date of purchase;
- Bill-to address:
- · Ship-to address:
- Number and description of units shipped;
- Name and telephone number of person to call, should contact be necessary;
- Reason for return and description of the problem.

Damage occurring during shipment is deemed the responsibility of the carrier, and claims should be made directly with the carrier.

For complete detailed information on warranty and on how to return your product, please visit www.clarityproducts.com/returns.

# **REGULATORY INFORMATION**

#### PART 15 OF FCC RULES INFORMATION

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any

interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits of a Class B digital device, pursuant to Part 15 of FCC rules. These limits are designed to provide reasonable protection against harmful interference in residential installation.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation; if this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, you are encouraged to try to correct the interference by one of the following measures:1. Where it can be done safely, reorient the receiving television or radio antenna.

2. To the extent possible, relocate the television, radio or other receiver with respect to the device.

**WARNING:** To maintain the compliance with the FCC's RF exposure guideline, place the base unit at least 20 cm from nearby persons.

For body worn operation, this device has been tested and meets the FCC RF exposure guidelines when used with the Clarity accessories supplied or designated for this product. Use of other accessories may not ensure compliance with FCC RF exposure guidelines.

**WARNING**: Changes or modifications to this equipment not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

# **REGULATORY INFORMATION**

### CAN ICES-3 (B)/NMB-3(B) (CANADA)

This Class B digital apparatus complies with Canadian ICES-003.

[The term "IC:" before the certification/registration number only signifies that the Industry Canada technical specifications were met.]

This digital apparatus does not exceed the Class B limits for radio noise emissions from digital apparatus set out in the interference-causing equipment standard entitled "Digital Apparatus," ICES-003 of the Canadian Department of Communications.

This device complies with Industry Canada license-exempt RSS standard(s).

Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.