


COMFORT
DUETT

A quality product from Sweden

TELEPHONE-KIT

The telephone kit connects a stationary (a fixed landline) telephone to the base unit of the Duett. The Comfort Duett amplifies the sound from the phone when it is placed into the base unit. You speak as usual through the telephone handset.

Using the Comfort Duett for telephone amplification requires no extra wires attached to the communicator. It automatically switches back to the prior communication mode, T-coil or MIC, when lifted out of the base.

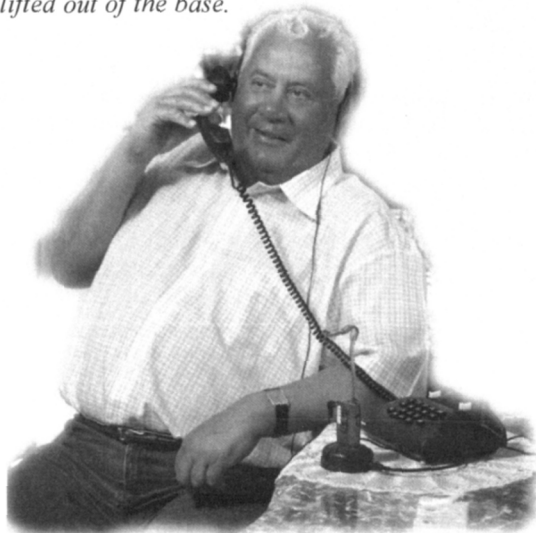

Before using the Telephone Kit, you should read the User Manual for Comfort Duett and the Limited Warranty.

 COMFORTaudio

Comfort Audio Inc., Customer Service, 5161 Overland Ave,
Culver City, CA 90230, Phone: (310) 945 3048

COMFORT
DUETT

A quality product from Sweden

TELEPHONE-KIT

The telephone kit connects a stationary (a fixed landline) telephone to the base unit of the Duett. The Comfort Duett amplifies the sound from the phone when it is placed into the base unit. You speak as usual through the telephone handset.

Using the Comfort Duett for telephone amplification requires no extra wires attached to the communicator. It automatically switches back to the prior communication mode, T-coil or MIC, when lifted out of the base.



 COMFORTaudio

Installation guide:


Place the base unit by the stationary phone. Alternatively, the base unit can be wall mounted. Plug in the power cord into a working electrical outlet and to the DC power jack in the base unit.

The telephone kit includes 3 different cables. The splitter cable, (A), is to be used between the telephone and the handset, with the clip end into the base of the telephone. The coiled handset cord clips into one of the two slots on the A-cable.

The other two cables, nearly identical, are marked with 1 and 2 resp. Try which of the cables, 1 or 2, offers the best sound quality; it varies by different telephones.



Step-by-step instruction:

1) Disconnect the coiled handset cord from the telephone base. To disconnect the cord, press down the small clip before pulling gently, see picture.

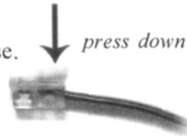


2) Connect the splitter cable, A, to the jack on the telephone from where the coiled cord was just removed, see picture. A "click" should be heard when the cable is connected properly.

3) Connect the coiled cord from the handset into one of the two jacks on the splitter cable that was just connected to the telephone, see picture below.

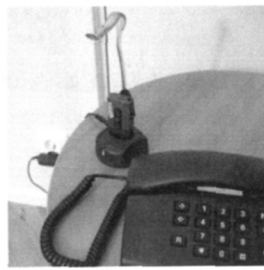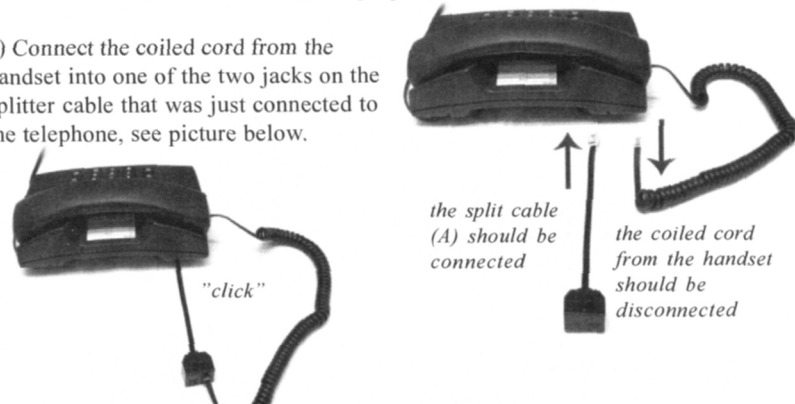

4) Attach the clip end of cable (1) in the free jack on the splitter cable (A) and the other end in the jack on the right side of the base unit.

5) Place the Comfort Duett in the base unit. Slide the mode selector to the top position (MIC/TV-mode) and place the head/earphones in a comfortable position. Lift the handset and adjust the volume on Comfort Duett to a comfortable level.

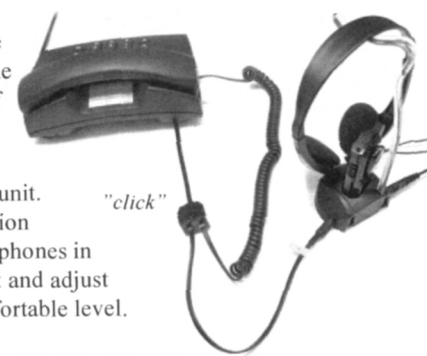

6) Redo steps 4 and 5 again but replace cable (1) with cable (2). Compare which of the cables gives the best sound quality and use that one.

Comfort Duett is now ready to use with your telephone. You talk in the handset as usual, but listen with the head/earphones. When using Comfort Duett to amplify a phone call, it is also being charged.

Hearing Aids

Wearers of Hearing Aids equipped with T-coil can use our stylish and lightweight neck loop (SKU CD0730) instead of head/earphones. The call will then be heard in the hearing aid when set to T-mode.

Alternative set-up

It is possible to connect the telephone directly to Comfort Duett instead of connecting via the base unit. Cable (1) or (2) is then plugged directly into the jack at the top of Comfort Duett (no. (16) in the User Manual). Comfort Duett will not be charged when using this connection.

Troubleshooting when Comfort Duett is not amplifying the call:

- Make sure all cables are attached properly to the telephone and the base unit.
- Make sure that the mode selector is in the top position and that the green LED on the base unit is lit.
- Make sure Comfort Duett is placed correctly in the base unit.
- Make sure that the head/earphones are connected to Comfort Duett.
- Make sure that the volume is correctly adjusted.
- Make sure that the maximum volume control is correctly adjusted, please see the Comfort Duett User Manual (The manual can also be found at our website).