CentralAlert[™]

**Model CA-360
Visual Alert System**

Operating Guide

Model
CA-360

Table of Contents

1. Table Of Contents.....	2
2. Important safety instructions.....	3-4
3. Package contents.....	4
4. Feature Identifications.....	5-6
5. Connection, set-up & testing.....	7-8
6. Bed Shaker indicators.....	8
7. Select 12/24 hour mode.....	8-9
8. Set the clock.....	9
9. Set the alarm time.....	9-11
10. Telephone function test.....	11
11. SOS/Test (help) function.....	11
12. Direct connect to an external lamp module, model CA-LX	11
13. Clear all accessories (Factory Default).....	12
14. Direct connect to a NOAA (Weather Alert) radio or external alarm system.....	12
15. Installation.....	13
16. Doorbell & test.....	13
17. Replacing doorbell battery.....	14
18. Adding extra wireless sensor accessories to the system.....	14-15
19. Convert a CA-360 from “Master Unit” into a “Remote Receiver”.....	16
20. Convert a CA-360 from “Remote-Receiver” into a “Master Unit”.....	17
21. Detection of Unregistered Audio Sensors CA-AX (Optional).....	18-19
22. Battery Condition Indicators.....	19-20
23. Technical Specifications.....	21
24. Display Messages.....	22
25. FAQ.....	23-25
26. Accessories for the CentralAlert™ Master Unit.....	26
27. Warranty service information.....	27
28. FCC information for USA customers only.....	28-30

Thank you for purchasing this **CentralAlert™** Model CA-360 Tabletop Notification System. Please read this operating guide very carefully before use and keep it for future reference. If you have any questions, please feel free to contact our Customer Care Department.

Toll-Free Help Line: 866.376.9271 (8AM-5PM PST, M-F)

IMPORTANT SAFETY INSTRUCTIONS:

Please read these instructions carefully before using the CA-360 Table Top Notification System

When using the CentralAlert™ Model CA-360, in order to reduce the risk of fire, electric shock or injury to persons, basic safety precautions should always be followed, including the following:

1. Read all instructions before using this product.
2. Do not mount or place Master Unit on a metal surface. Metal can reduce wireless signal transmission.
3. The CA-360 Master Unit's clock uses rechargeable batteries. **DO NOT** use non-rechargeable batteries as charging of this type can cause battery leakage.

Disposal of batteries:


Please dispose of the batteries properly, following any local regulations. **DO NOT** dispose of the system's batteries in the household trash. **DO NOT** incinerate. Take the equipment to a nearby recycle center.


4. Always disconnect phone cord from wall jack before changing/ installing batteries.
5. To reduce injury risk, close supervision is necessary when appliance is used near children.
6. Do not make contact with moving parts or exposed metal surfaces.
7. Only use attachments/ accessories recommended/sold by the manufacturer.
8. Do not use outdoors. Do not use near water.
9. To disconnect, turn all controls to "Off" position, then remove plug from outlet.

IMPORTANT SAFETY INSTRUCTIONS (con't):

10. Do not unplug the unit by pulling the cord - grasp plug, not cord.
11. Unplug unit from outlet when not in use/before servicing or cleaning.
12. Do not operate this system with a damaged cord or plug; do not operate any product after it malfunctions, is dropped or damaged in any manner. Return product to nearest authorized service facility for examination, repair, or electrical/mechanical adjustment.
13. Avoid using telephone products during electrical storms. There is an electric shock risk from lightning.
14. Do not use a telephone to report a gas leak while in the vicinity of the leak.
15. Specifications of this product are subject to change without notice.

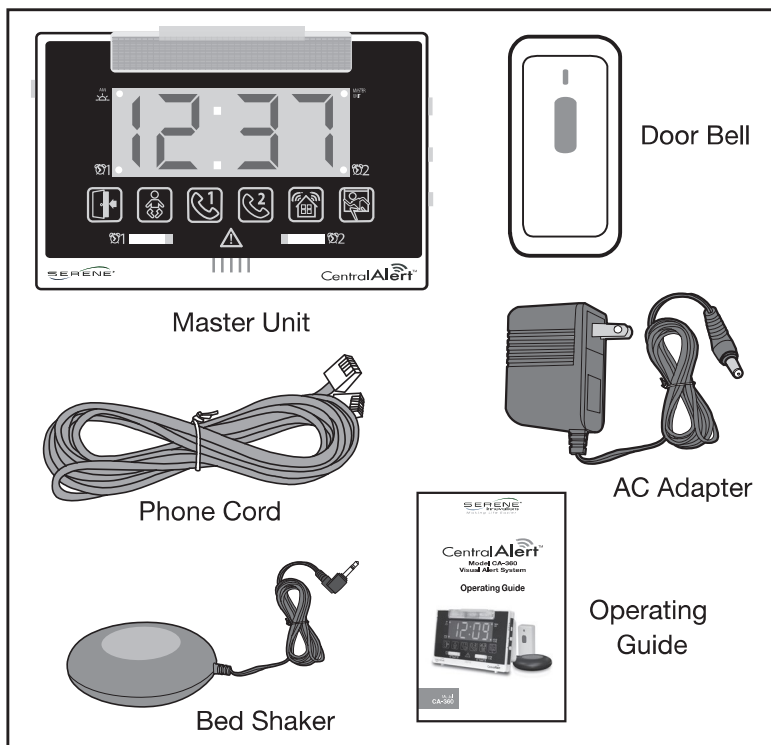
PACKAGE CONTENTS:

FEATURE IDENTIFICATIONS: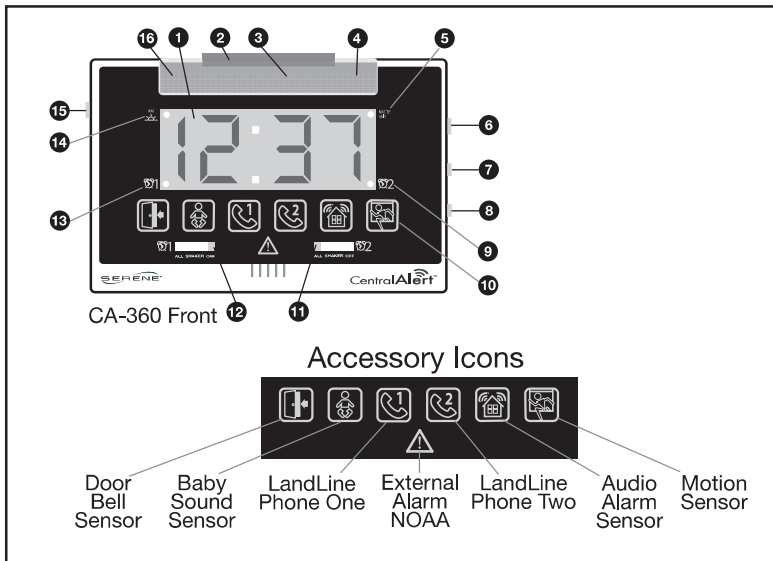



1. Clock display (1.25" tall)-12/24 hour format with auto-dim
2. Reset/Snooze button
3. Flasher
4. Bed Shaker 2 indicator (Model CA-360v2). Indicator **ON**-Bed shaker connected. Indicator **OFF**-Bed shaker not connected.
5. Master Unit indicator
6. Set switch
7. Hour
8. Minute
9. Alarm 2 "**ON**" indicator
10. Accessory icons (7)
11. Alarm 2 wake-up option switch
12. Alarm 1 wake-up option switch
13. Alarm 1 "**ON**" indicator
14. A.M. indicator
15. Hi – Low – Off (speaker)
16. Bed Shaker 1 indicator (Model CA-360v2). Indicator **ON**-Bed shaker connected. Indicator **OFF**-Bed shaker not connected.

FEATURE IDENTIFICATIONS (con't):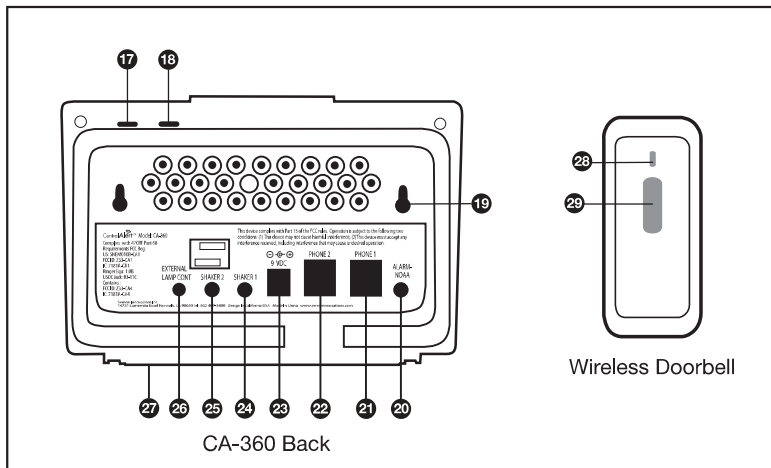

17. Motion sensor (**ON/OFF** button)
18. **SOS** /test button
19. Holes for wall mount
20. Alarm/**NOAA** input jack (for connection to weather alert radio and external alarm system)
21. Phone 1 jack for landline phone /FAX/ Video phone
22. Phone 2 jack for landline phone /FAX/ Video phone
23. AC Adapter jack (9VDC, 1000mA)
24. Shaker 1 jack
25. Shaker 2 jack
26. External lamp module connection jack (lamp module is optional accessory)
27. Battery compartment (underneath). Batteries **NOT** included
28. Transmit indicator (doorbell)
29. Doorbell push button (doorbell)

CONNECTION, SET-UP & TESTING: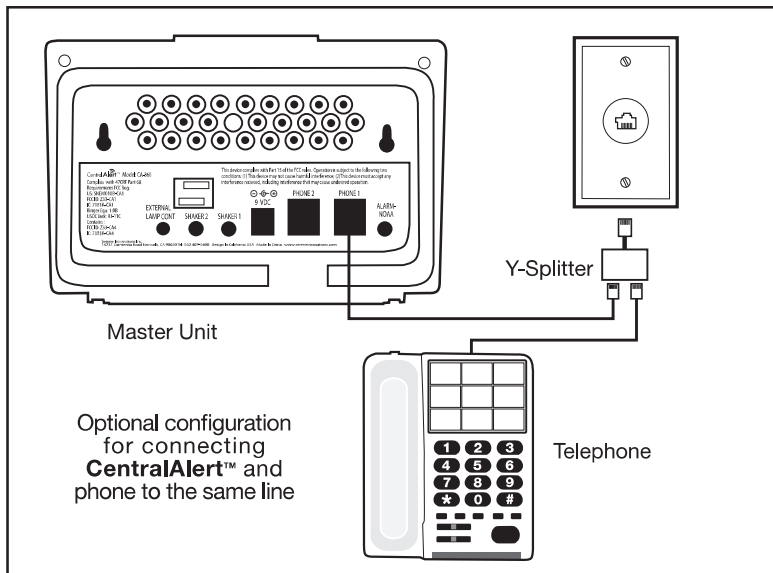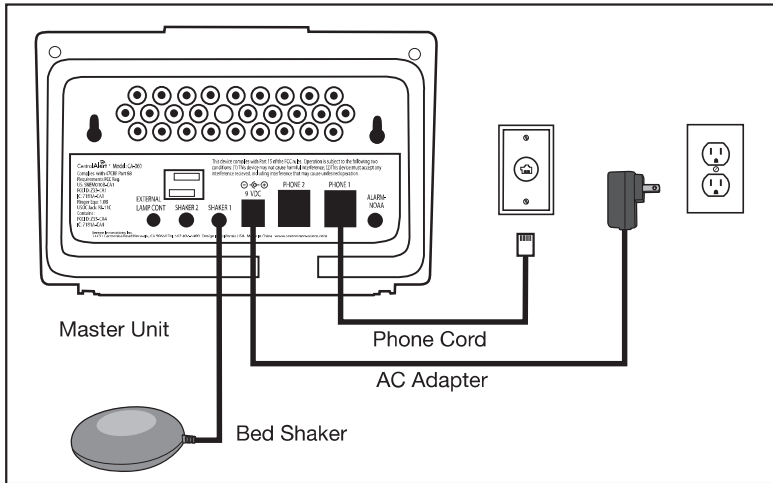

1. Install four AA size rechargeable NiMH batteries (user provided) in battery compartment at bottom of the CA-360 Master Unit.

Note: The CA-360 Master Unit works without batteries. Batteries may be inserted later against power outage.

CONNECTION, SET-UP & TESTING (CONT.):

Warning: Do not use regular batteries as charging of this type can cause battery leakage and damage to the CentralAlert™ Master Unit.

2. Plug AC adapter into nearest electrical outlet and connect adapter plug to **CentralAlert™** Master Unit as shown.
3. Connect bed shaker to **SHAKER-1** output jack. The **SHAKER-1** indicator will be lit (CA-360v2).

Bed Shaker Indicators (CA-360V2) :

There are two (2) bed shaker indicators. One is located on the left side and the other is located on the right side of the flasher, representing the **SHAKER-1** (left), or the **SHAKER-2** (right) connection status. When a bed shaker is connected to the Master Unit through the back input jack, the corresponding indicator light will be lit.

4. Connect phone cord to **PHONE-1** outlet at the back of Master Unit, then plug the other end into wall phone jack.
5. Set the **SPEAKER** (on left side of unit) to desired volume setting (**Hi-Lo-Off**).
6. Press **SOS/Test** button on top of unit to test. Verify that alert outputs (speaker, flasher, shaker, and display) last for 30 seconds, then stop. (Press **SNOOZE/RESET** button to stop early).
7. Verify that flasher can be toggled **ON** and **OFF** using **SNOOZE / RESET** button.
8. Verify “Master Unit” indicator is always “**ON**.”

Select 12/24 Hour Mode:

1. The clock may be set for either 12 or 24 hour mode (U.S. or Europe)
2. The factory default clock setting is in 12-hour mode.
3. To change the mode to 24-hour mode, slide the **SET** switch to **TIME** position, and then press and hold **SOS/TEST** button for 6 seconds. The **AM** indicator will go off if the clock is in the morning hours.

CONNECTION, SET-UP & TESTING (CONT.):

Select 12/24 Hour Mode (CONT.):


- Repeat Step 3 to change setting back to 12-hour mode.
The **AM** indicator will be **ON** if the clock is in the morning hours.
- Slide **SET** switch to **RUN** position for normal clock operation.

Set the Clock:

- Slide **SET** switch (on right side of Master Unit) to **TIME** position and push **HR & MIN** buttons to set time. Each press advances one count. Pressing and holding button advances count continuously.
- After completing time setting, slide **SET** switch to **RUN** position or normal clock operation.


Set the Alarm Time:

- Slide **SET** switch (on right side of Master Unit) to **Alarm-1** (clock icon-1) position and set alarm time using **HR** and **MIN** buttons.
- After completing the setting, slide **SET** switch to **RUN** position for normal clock operation.
- Select the front panel wake-up option switch on **SHAKER** or **ALL** setting. **SHAKER** will wakeup user via the “bed shaker and flasher only.” **ALL** means waking up user via “bed shaker, flasher and speaker.”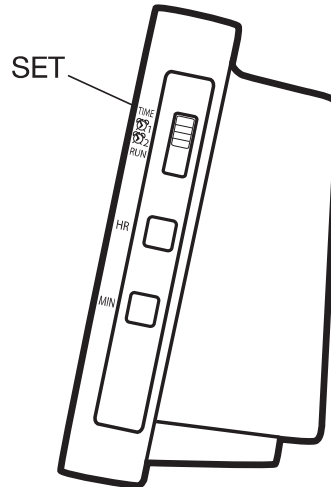

CONNECTION, SET-UP & TESTING (CONT.):

Set the Alarm Time (CONT.):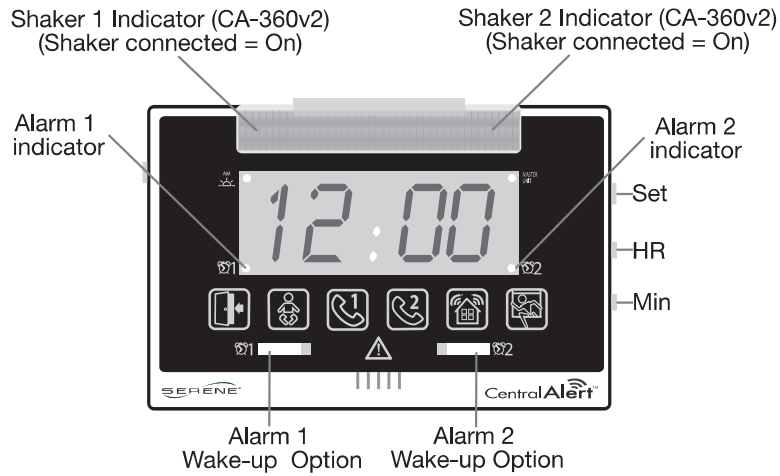

- Repeat the above steps for the second alarm settings by selecting the **Alarm-2** (clock icon-2) position. The Master Unit will show “AL 1” (or “AL 2”) when alarm goes off.

(**Note:** Alarm 1 or 2 indicators will stay on when the alarm 1 or 2 switch **ALL** or **SHAKER** position is selected.)

- The alarm clock activates for 45 seconds at set time, then stops for 5 minutes. This pattern will repeat for an entire one (1) hour.
- While alarm clock is being activated, a quick press of **RESET/SNOOZE** button allows for a 5-minute snooze.
- A long (2-second) press of the same **RESET/SNOOZE** button turns off the alarm until the next day. The display will show “Stop”.
- Slide the front panel wake-up option switch to **OFF** position to permanently turn off the alarm function.

CONNECTION, SET-UP & TESTING (CONT.):

Set the Alarm Time (CONT.):

Note: **SHAKER-1** and **SHAKER-2** output jacks correspond to Alarm-1 and Alarm-2 individually, but **BOTH** jacks will be active during alerts. For alarms, **Alarm 1** will vibrate only bed shaker 1. **Alarm 2** will vibrate only bed Shaker 2.

Telephone Function Test:

1. Place a call to the phone number corresponding to **PHONE-1**.
2. Verify that all alerting outputs respond correctly while phone is ringing.
3. Verify that by answering the call (from any phone), all alerting outputs stop immediately.
4. The **PHONE-2** jack is available for connection to a second phone line. This second line can be a regular phone line, FAX line, or video-phone line.

SOS/TEST (Help) Function:

Pressing the **SOS/TEST** button or the **EMERGENCY HELP** button on the Model CA-SOS, sends a wireless signal to activate **SOS** alert outputs on all CentralAlert™ receiver units in your system.

Direct Connect to an External Lamp Module, Model CA-LX:

The CentralAlert™ Master Unit can control an external lamp via an optional, external lamp module (Model CA-LX). A user can control the lamp **ON/OFF** using the **RESET/SNOOZE** button on top of the CentralAlert™ Master Unit. The operation of this lamp is synchronized with the flasher.

Note: The external lamp must be left in the **ON** position.

CONNECTION, SET-UP & TESTING (CONT.):**Clear All Accessories From Master Unit (Factory Default):**

1. Power up the Master Unit and place **SET** switch to the **RUN** position.
2. Press and hold the “**SOS/Test**” button on the Master Unit for 6 seconds until display reads “**LEAr**”. Now press and release “**SOS/TEST**” button 18 times in quick succession to clear its internal memory.
3. When successful, the unit will sound a beep and the display will show “**dEL**” momentarily.
4. Press **RESET/SNOOZE** button to exit the mode.

Direct Connect to a NOAA (Weather Alert) Radio or External Alarm System

CentralAlert™ is compatible with NOAA radios that offer a 3.5mm alert output jack. (Midland Model WR-120, RadioShack Model 12-382; First Alert Model WX-150; Sangean CL-100 may offer an alert jack feature).

Please note that setting up a **NOAA** radio for proper operation requires a good understanding of how a weather radio works.

Please consult with the radio manufacturer for technical help. Neither Serene Innovations, Inc., nor its dealers are responsible for any consequences relating to the purchase or the use of these radios.

The “**Alarm/NOAA**” Input jack is also compatible to an alarm system where a “Normal Open” relay contact is available. When properly connected, a momentary “Close” or “Short” on the relay contact activates the triangular Warning Icon and all alerting outputs. Please consult with an alarm installer on alarm connections.

CONNECTION, SET-UP & TESTING (CONT.):


Installation:

1. Connect one end of a 3.5mm plug cable (user provided) to the “**Alarm/NOAA**” input jack.
2. Connect the other end of the plug cable to a NOAA radio alert output jack.
3. Verify that the alert output from the radio can trigger during a National Weather Service test alert broadcast. Contact the National Weather Service for information about test alert dates and time.

Doorbell and Test:

The CentralAlert™ Model CA-360 comes complete with a wireless, weather-resistant doorbell.

1. Activate doorbell button 50 ft from CentralAlert™ Master Unit. Confirm that all alerting outputs respond correctly and last for 45 seconds. Verify that display shown “**dr -n**”, where n is the doorbell number. If not, see “Adding extra wireless accessories to the system” section in the manual.
2. Separate mounting plate from doorbell and install plate near door using two (2) wood screws, shown below.
3. Complete installation by snapping doorbell back onto mounting plate.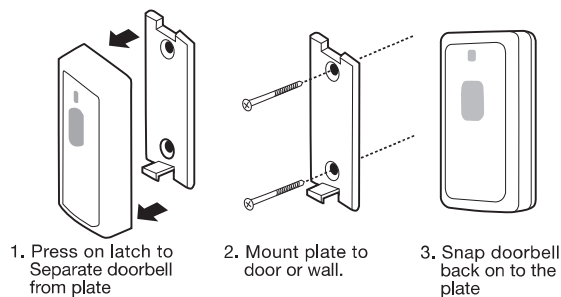


Note: DO NOT mount the doorbell on a metal surface as this weakens wireless transmission.

CONNECTION, SET-UP & TESTING (CONT.):

Replacing Doorbell Battery

The battery (CR2032, 3V lithium) in the doorbell normally lasts 8 to 16 months (depending on how often it is used). This battery needs to be replaced if the doorbell does not operate consistently or if its transmit indicator becomes dim.

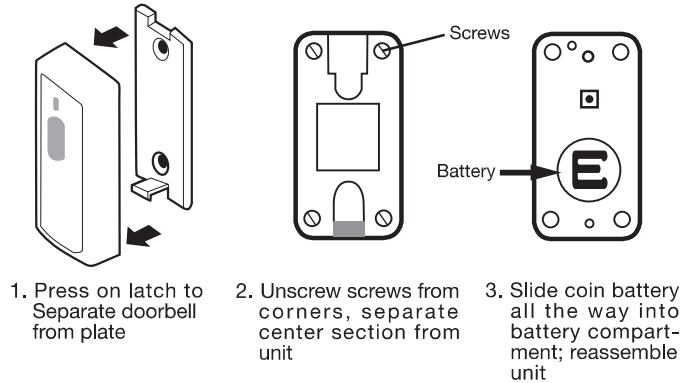Note: The battery is in the correct position if the lettering on the battery is face up.

Adding Extra Wireless Sensor Accessories to the system:

1. A new wireless sensor (or an extra wireless doorbell) must be learned (registered) into the Master Unit before it can be recognized.
2. With **SET** switch in the **RUN** position, press and hold the **SOS/TEST** button for 6 seconds until it beeps and display flashes "**LEAR**", indicating the Master Unit is ready to learn (register) a new device.
3. While the Master Unit is in the Learn mode, press and hold the new accessory's test button for about 6 seconds until the Master Unit beeps twice and the display shows "**xxxx**", confirming registration is successful (see table below).

CONNECTION, SET-UP & TESTING (CONT.):**Adding Extra Wireless Sensor Accessories to the system (CONT.):**

4. Press the **RESET/SNOOZE** button to end the Learn process.
5. Test the new wireless accessory by pressing its test button.
If the Master Unit reacts and shows the corresponding accessory display, the learning was successful. If not, repeat the steps above to learn the accessory again.

Wireless Accessory	Display (n= number assigned)	Maximum Number In a System
CA-AX	AS- <i>n</i>	10
CA-BX	BS- <i>n</i>	3
CA-CX	Sub	1
CA-DB	dr- <i>n</i>	4
CA-DX, CA-DXH	dr- <i>n</i>	Any combination
CA-MX	SS- <i>n</i>	3
CA-NOAA	Sub	1
CA-PX	Sub	10
CA-RX	Sub	10
CA-SOS	Sub	4
CA-AX	AS 0	Special display – indicates a nearby CA-AX (unregistered) signal has been detected. This detect function is factory preset to OFF . Refer to page 18 in this operating guide to enable this function.

CONNECTION, SET-UP & TESTING (CONT.):**Convert a CA-360 from “Master Unit” into a “Remote Receiver”:**

This is a two-step process: The first step **(A)** below is to clear any accessory devices that have been learned into the unit. The second step **(B)** changes the unit into a “Remote Receiver”. Please follow these steps. If required, please contact Customer Care Service Department for assistance.

(A). Clear Internal Memory:

1. Power on the Master Unit and place **SET** switch to the **RUN** position.
2. Press and hold the “**SOS/Test**” button on the unit for 6 seconds until display reads “**LEAR**”. Now press and release “**SOS/TEST**” button 18 times in quick succession to clear its internal memory.
3. When successful, the unit will sound a beep and the display will show “**dEL**” momentarily.
4. Press **RESET/SNOOZE** button to exit the mode.

(B). Change The Master Unit into a Remote Receiver:

1. Slide the **SET** switch on the Master Unit to the **TIME** position.
2. Press and hold “**MOTION**” button for 6 seconds until display reads “**Sub**” (subordinate). The Master Unit is now a “Remote Receiver”. Verify its “Master Unit” indicator is **OFF** (upper right corner of display).
3. Now slide the **SET** switch back to the **RUN** position.
4. A “Remote Receiver” is an accessory device. It must be learned (or registered) to a “Master Unit” before it can be recognized. See earlier sections in this manual for instructions as how to register an accessory device.
5. When implemented correctly, the combined system can only have one “Master Unit” and up to 10 combinations of “Remote Receivers”.

CONNECTION, SET-UP & TESTING (CONT.):**Convert a CA-360 from a “Remote Receiver” into a “MasterUnit”:**

This is a two-step process: The first step **(A)** below is to clear any learned accessory devices in the unit. The second step **(B)** changes the unit back into a “Master Unit”. Please follow these steps, or if required, please contact Customer Care Service Department for assistance.

(A). Clear Internal Memory:

1. Power on the “Master Unit” and place **SET** switch to the **RUN** position.
2. Press and release “**SOS/TEST**” button 18 times in quick succession to clear its internal memory.
3. When successful, the unit will sound a beep and the display will show “**dEL**” momentarily.
4. Press **RESET/SNOOZE** button to exit the mode.

(B). Change into a “Master Unit”:

1. Slide the **SET** switch to the **TIME** position. The clock will blink.
2. Press and hold “**MOTION**” button for 6 seconds. The clock will stop blinking temporarily, the “Master Unit” indicator will be **ON**, and the unit will sound a beep. The clock will blink again.
3. Now slide the **SET** switch back to the **RUN** position.
4. A “Master Unit” is the main receiver of a system. A system cannot have two (2) “Master Units”. All accessories to be used in the system must be learned (or registered) to the “Master Unit” before it can be recognized. See earlier sections in this manual for instructions as how to register accessory devices.
5. When implemented correctly, an alerting system can only have one “Master Unit” and up to 10 combinations of “Remote Receivers”.

NEW FUNCTIONS FOR CA-360v2 :

Detection of Unregistered Audio Sensors CA-AX (Optional):

The CentralAlert™ Model CA-360v2 is designed with a new feature that allows the user to detect all surrounding and in range wireless audio alarm sound monitors (CA-AX, sold separately) that have not been learned (or registered) into this Model CA-360v2 Master Unit. This feature is particularly useful in dormitory or hotel installations.

When this feature is enabled and an activation of an unregistered CA-AX is detected, the CentralAlert™ Master Unit will display “AS 0”. A quick press on the **RESET/SNOOZE** button will cancel this alert within its own system, but **NOT** to other adjacent CentralAlert™ systems.

Please Note: The factory default setting on this feature is **OFF**.

To Enable this setting:

- a. Disconnect all power including (AC adapter and batteries (if used)) from the CA-360v2 Master Unit.
- b. Press and hold the **SOS/TEST** button while applying the AC power.
- c. The display will show a quick message “**AS ON**”.
- d. Proceed with the clock settings, and re-install the batteries (if used).
- e. The setting is enabled, and will be remembered even if all power is removed.

This unique feature alerts the user on CA-360v2 to the activation of a CA-AX that was in an adjacent room, or in the hallway (in range), without the need to learn (or register) the CA-AX.

NEW FUNCTIONS FOR CA-360v2 (CONT.):

Detection of Unregistered Audio Sensors CA-AX (Optional) (CONT.):

To Disable this setting:

- a. Disconnect all power (AC adapter and batteries (if used)) from the CA-360v2 Master Unit.
- b. Press and hold the **MOTION** button while applying the AC power.
- c. The display will show a quick message “**AS OF**”.
- d. Proceed with the clock settings, and re-install the batteries (if used).
- e. The setting is disabled, and will be remembered even if all power is removed.

Battery Condition Indicators: (New)

The new CentralAlert™ Model CA-360v2 is designed with a new feature to alert users to a low battery condition:

A. CA-360v2 Master Unit running on Batteries:

When the CA-360v2 is in battery mode, the display will be turned **OFF**. A quick press on the **RESET/SNOOZE** button will display the clock for 3 seconds. A second press on the **RESET/SNOOZE** button within the 3 seconds will turn **ON** the flasher light. Press the **RESET/SNOOZE** again to turn **OFF** the flasher. The clock will be redisplayed when the AC adapter is used.

B. Low Battery Condition on CA-360v2:

When the CA-360v2 Master Unit batteries are running low, the CA-360 flasher will blink once every 3 seconds. All alerting functions are operational during the low battery condition. Check and re-apply the AC power adapter to the CA-360v2 Master Unit as soon as possible.

NEW FUNCTIONS FOR CA-360v2 (CONT.):

Battery Condition Indicators: (New) (CONT.):

When the batteries on the CA-360v2 Master Unit are totally depleted, the flasher will stop blinking, and the CA-360v2 Master Unit will stop operation. Apply the AC power adapter to the CA-360, and the flasher will start flashing, indicating it is recovering from the low battery state, but the unit will not start. The clock will not display until the batteries have returned to a minimum charged level. This process may take 5-10 minutes. Once the clock is displayed, reset the clock and the CA-360v2 Master Unit is ready to use. All previous learned accessories learned (or registered) with the CA-360v2 are still intact.

C. Low Battery Signals from registered CA-AXv2:

When a low battery signal from a registered CA-AXv2 is detected, the CA-360v2 Master Unit will display a “**batt**”, “**-nn**” repeatedly to alert the user that a registered CA-AXv2 audio sensor's battery is running low. Change a new set of fresh batteries into that CA-AXv2 to stop the low battery signal transmitting from that CA-AXv2.

To temporarily stop the low battery display, press and hold the **RESET/SNOOZE** button for more than 2 seconds. The display will return to the normal clock display. When it receives another low battery signal from a CA-AXv2 again, the “**batt**” display will re-appear.

Note: The CA-AXv2 transmits its low battery signal once every 30 minutes.

TECHNICAL SPECIFICATIONS:**(Model CA-360v2 Master Unit)**

Technology.....	2.4 GHz Smart-Code
Phone Ringing Input.....	>40 vrms, 20 to 65 Hz
Phone jack type.....	RJ-11
Bed-shaker output jack.....	9VDC, 150mA
Alarm/NOAA Input jack.....	Input impedance greater than 5.6K Ohm,15VDC Max. input (active low).
External Lamp Control output jack.....	9VDC, 150mA
Back-up batteries.....	4 AA size NiMH batteries (user provided) Recommended >1000mAh capacity.
Doorbell battery.....	(CR2032 3-Volt Lithium) Lasts 8-16 months (depending on how often it is used)
AC Adapter output.....	9VDC, 1000mA
Operating temperature.....	10° to 30° C
Storage temperature.....	-10° to 40° C

Specifications are subject to change without notice.

DISPLAY MESSAGES :

	DISPLAY	INDICATIONS
1	bOSS	Start up display, unit is a "Master" unit
2	Sub	<ul style="list-style-type: none"> • Set the CA-360 as a subordinate receiver; or • Learn a receiver (CA-RX, CA-PX, CA-CX, CA-360)
3	dr -n	A doorbell/door knocker unit number (CA-DB, CA-DX)
4	AS -n	An audio sensor unit number (CA-AX)
5	bAtt -n	Low battery warning number from paired CA-AX's
6	bS -n	A baby sound sensor unit number (CA-BX)
7	CELL	A cellphone sensor (CA-CX)
8	NOAA	A weather alert radio input or sensor (CA-NOAA)
9	SOS	A SOS signal input activated (CA-SOS)
10	LEAR	Master unit is in Learn mode
11	AL 1	Clock alarm 1 time setting
12	AL 2	Clock alarm 2 time setting
13	ALL	Speaker, shaker, and flasher are all selected
14	SHA	Shaker and flasher are selected
15	OFF	Alarm is switched to "OFF"
16	Stop	<ul style="list-style-type: none"> • Stop alarm clock snooze function; or • Stop low battery status from registered CA-AX
17	PH 1	Phone 1 ringing
18	PH 2	Phone 2 ringing
19	SEt	Set Time
20	rUN	Normal clock RUN mode
21	ON	Motion sensor input is Enabled (CA-MX)
22	OFF	Motion sensor input is Disabled (CA-MX)
23	dEL	Delete or Clear all accessories learned (registered)
24	AS ON	Unit will detect any unregistered CA-AX in range
25	AS OF	Unit will NOT detect any unregistered CA-AX (default)
26	AS 0	An activation of an unregistered CA-AX is detected

FAQ :**1. Must I install batteries in the CentralAlert™ Master Unit ?**

Ans: Batteries are for power outage back-up. The unit works with AC adapter and batteries are optional. It is strongly recommended that you install batteries.

2. What type of batteries does it need?

Ans: Please install rechargeable NiMH batteries only. Use of regular or non-rechargeable batteries will cause battery leakage, damage unit and void the product warranty.

3. What is the battery back-up time on the CA-360v2 Master Unit ?

Ans: About 7 days (normal operation, infrequent alerts)

4. Is it alright to place the CentralAlert™ Master Unit on a metal surface?

Ans: No! Only place it on a non-metallic surface. A metal surface attenuates wireless signal transmission and reduces its operating range.

5. Is the wireless doorbell water-resistant?

Ans: Yes, it is water resistant because it is protected with a rubber gasket. However, do not submerge in water.

6. How long does the doorbell battery last?

Ans: About 8 to 16 months (depends on how often you use it). Replace battery when its transmit indicator becomes dim or if its operation becomes less consistent.

FAQ (CONT.):**7. How many doorbell units can my CentralAlert™ system support?**

Ans: It can support a total of 4 units in any combination of doorbell (Model CA-DB), and door knocker sensor (Model CA-DX).

8. How about other accessories?

Ans: Refer to the table in the “Adding Accessories Section”. The system can support up to 10 Remote-Receivers (CA-RX), 10 Personal pagers (CA-PX), 10 audio sensors (CA-AX, CA-AXv2), 4 SOS pendants (CA-SOS), 3 Baby-Sound sensors (CA-BX), 1 landline/cell-phone sensor (CA-CX), and 1 NOAA sensor (CA-NOAA).

9. I just bought an extra doorbell for my CentralAlert™ system, but it did not work. What do I need to do?

Ans: All new accessory devices must be learned or registered by a CentralAlert™ Master Unit before they can be worked or recognized. See the earlier sections in this manual for instructions as how to register an accessory device.

10. I bought four CentralAlert™ systems for my home. Can I combine them into a single system?

Ans: Yes you can. You need to convert three of the CentralAlert™ units from Master Unit into a “Remote-Receiver” so they can be learned (or registered) by the remaining Master Unit. (See the section on how to convert CA-360 into “Remote Receivers”).

FAQ (CONT.):**11. How do I reset low battery message “batt -xx” on CA-360?**

Ans: Press and hold “**SNOOZE/RESET**” button for 2 seconds. (Note that xx identifies device number)

12. What caused this low battery message to come on?

Ans: This message is only associated with audio sensor CA-AX. It indicates CA-AX battery is either low or not being turned on, or it may be placed too far from CA-360. Turn on its power or move the unit closer to CA-360 usually solve the issue.

13. I have difficulty in registering an accessory to my CA-360 system. What should I do?

Ans: Make sure you press and hold the button on the accessory for at least 6 seconds when registering, or until you hear CA-360 beeps twice for confirmation.

14. I have a large home, can I use two or more CA-360's to increase coverage area?

Ans: Yes, because CA-360 has built-in repeater function. Each CA-360 can relay the signal it received to the next unit. This increases effective coverage area.

15. Why is my CA-360v2 displaying “AS 0” even though none of my CA-AX's are activated?

Ans: Your CA-360v2 has enabled the feature “Detection of Unregistered Audio Sensors CA-AX”. You can disable it by following the steps described earlier in that section.

**ACCESSORIES FOR THE CENTRALALERT™
MASTER UNIT:**

MODEL	DESCRIPTION
CA-AX	Audio Alarm Sensor
CA-BX	Baby Sounds Sensor
CA-CX	Landline/Cell Phone Ringer/Flasher Sensor Transmitter
CA-DB	Weather Resistant, Wireless Doorbell
CA-DX	Door Knocker Sensor
CA-DXH	Door Hanging Door Knocker
CA-LX	Lamp Flasher Connector
CA-MX	Motion Detector
CA-NOAA	Emergency Weather Warning Sensor
CA-PX	Wearable Personal Receiver
CA-RX	Portable Remote Receiver
CA-SOS	Emergency Help Button Sensor
BS-100	Bed Shaker
CA-BP	Four (4) AA Size NiMH Rechargeable Batteries

WARRANTY SERVICE INFORMATION:

Your **CentralAlert™ Model 360** comes with a one year limited warranty. We warrant during this warranty period from the date of purchase to the original consumer that these products will be free from defects in materials and workmanship under intended home use. In the event that these products fail to function properly within one year of original purchase due to defects in materials or workmanship, return the unit (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to Serene Innovations, Inc.. Serene Innovations, Inc. will either repair or replace the unit (with a refurbished unit or unit of equal condition) and return it to you (using UPS/USPS ground shipping) at no cost to you, if the unit is returned within 30 days of purchase. If the unit is returned after 30 days of purchase but within the warranty period, there will be no cost to you (there will be a nominal additional shipping charge if it is shipped to out-side the 48 Continental U.S. states).

This warranty does not apply to any product that has been accidentally damaged due to: abuse, misuse, negligence, use on electrical frequency or voltage other than marked on product and/or described in this manual. Defects or errors caused by unauthorized alterations, repairs and/or tampering are also not covered by this warranty.

Incidental or Consequential Damages:

Neither Serene Innovations, Inc. nor the retailer, dealer or selling distributor has the responsibility for any incidental or consequential damages, including, without limitation, commercial loss of profit or for any incidental expenses, loss of time or inconvenience. Some states do not allow exclusion or limitation of incidental or consequential damage, so the above limitation or exclusion may not apply to you

This warranty gives you specific legal rights. You may have other legal rights that vary from state to state. If a defect covered by this warranty should occur, promptly contact a Customer Care Representative by phone or log on to www.sereneinnovations.com to obtain a Return Merchandise Authorization number and shipping instructions before returning the product to us. Any shipment without a RMA will not be accepted and will be returned to you at your expense.

Any authorized returned product must be accompanied with proof of purchase and a brief description of the problem. For out of warranty repair and service, please contact our Customer Care Department for directions.

FCC INFORMATION FOR USA CUSTOMERS ONLY

This equipment complies with Part 68 of the FCC rules and the requirements adopted by the ACTA. On the bottom of this equipment is a label that contains, among other information, a product identifier of [US: XXXXXXXXX]. If requested, this number must be provided to the telephone company. If this equipment, causes harm to the telephone network, the telephone company will notify you in advance that temporary discontinuance of service may be required.

But if advanced notice isn't practical, the telephone company will notify the customer as soon as possible. Also, you will be advised of your right to file a complaint with the FCC if you believe it is necessary. The telephone company may make changes in this facility's equipment, operations or procedures that could affect the operation of the equipment. If this happens the telephone company will provide advanced notice in order for you to make the necessary modifications to maintain uninterrupted service. If you experience trouble with this equipment, disconnect it from the network until the problem has been corrected or until you are sure that the equipment is not malfunctioning. Please follow instructions for repair, if any (e.g., battery replacement section); otherwise do not alternate or repair any parts of device except as specified. Connection to party line service is subject to state tariffs. Contact the State Public Utility Commission, Public Service Commission or Corporation Commission for information. If the telephone company requests information on what equipment is connected to their lines, inform them:

- A. The telephone number that this unit is connected to,
- B. The ringer equivalence number [REN],
- C. The USOC jack required [RJ11C], and
- D. The FCC Registration Number [US: XXXXXXXXX].

Items (B) and (D) are indicated on the label. The ringer equivalence number (REN) is used to determine how many devices can be connected to your telephone line. In most areas, the sum of the RENs of all devices on any one line should not exceed five (5.0). If too many devices are attached, they may not ring properly.

This equipment complies with Part 68 of the FCC Rules.

On the equipment is a label that contains, along with other information, the FCC Registration Number and Ringer Equivalence Number (REN) for this equipment. If requested, this information must be provided to your telephone company.

This equipment uses the following Universal Service Order Codes ("USOC") jacks: RJ-11. Connection to the telephone network should be made by using standard modular telephone jack type RJ11.

The REN is useful to determine the quantity of devices you may connect to your telephone line and still have all of those devices ring when your telephone number is called. In most, but not all areas, the sum of the RENs of all devices connected to one line should not exceed five (5.0). To be certain of the number of devices you may connect to your line, as determined by the REN, contact your local telephone company to determine the maximum REN for your calling area.

If your telephone equipment causes harm to the telephone network, the telephone company may discontinue your service temporarily. If possible, they will notify you in advance. But if advanced notice is not practical, you will be notified as soon as possible. You will be informed of your right to file a complaint with the FCC.

FCC INFORMATION FOR USA CUSTOMERS ONLY (CONT.):

Your telephone company may make changes in its facilities, equipment, operations or procedures that could affect the proper functioning of your equipment. If they do, you will be notified in advance to give you an opportunity to maintain uninterrupted telephone service.

In the event this equipment should fail to operate properly, disconnect the unit from the telephone line. Try using another FCC approved telephone in the same telephone jack. If the trouble persists, call the telephone company repair service bureau.

If the trouble persists and appears to be with this unit, disconnect the unit from the telephone line and discontinue use of the unit until it is repaired. For repair or warranty information, please contact Serene Innovations, Inc. at 562-407 5400. Please note that the telephone company may ask that you disconnect this equipment from the telephone network until the problem has been corrected or until you're sure that the equipment is not malfunctioning.

There are no user serviceable parts in this equipment.

This equipment may not be used on coin service provided by the telephone company. Connection to party lines is subject to state tariffs. Contact the state Public Utility Commission, Public Service Commission for information.

If your home has specially wired alarm equipment connected to the telephone line, ensure the installation of this equipment does not disable your alarm equipment. If you have questions about what will disable alarm equipment, consult your telephone company or a qualified installer.

Part 15 of FCC Rules Information

This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. This equipment has been tested and found to comply with the limits of a Class B digital device, pursuant to Part 15 of FCC rules. These limits are designed to provide reasonable protection against harmful interference in residential installation.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, you may try one the following methods to correct the interference:

1. Where it can be done safely, re-orient the receiving television or radio antenna.
2. To the extent possible, relocate the television, radio, or other receiver, with respect to the cordless phone.
3. Connect the cordless phone AC adapter into a different electrical outlet.
4. Consult the dealer or an experienced radio/TV technician for help. Modification: Any modification not expressly approved by the manufacturer of this device could void the user's authority to operate the device.

FCC INFORMATION FOR USA CUSTOMERS ONLY (CONT.):**NOTICE: The Industry Canada label identifies certified equipment.**

This certification means that the equipment meets telecommunications network protective, operational, and safety requirements as prescribed in the appropriate Terminal Equipment Technical Requirements document(s). The Department does not guarantee the equipment will operate to the user's satisfaction.

Before installing this equipment, users should ensure that it is permissible to be connected to the facilities of the local telecommunications company. The equipment must also be installed using an acceptable method of connection. The customer should be aware that compliance with the above conditions may not prevent degradation of service in some situations. Repairs to certified equipment should be coordinated by a representative designated by the supplier. Any repairs or alterations made by the user to this equipment, or equipment malfunctions, may give the telecommunications company cause to request that the user disconnect the equipment. Users should ensure, for their own protection, that the electrical ground connections of the power utility, telephone lines, and internal metallic water pipe system, if present, are connected together. This precaution may be particularly important in rural areas.

Caution: Users should not attempt to make such connections themselves, but should contact the appropriate electric inspections authority, or electrician, as appropriate. The REN for this amplified telephone is stated on the IC regulatory label located on the bottom of the product.

NOTICE: The Ringer Equivalence Number (REN) assigned to each terminal device provides an indication of the maximum number of terminals allowed to be connected to a telephone interface. The termination on an interface may consist of any combination of devices, subject only to the requirement that the sum of the Ringer Equivalence Numbers of all the devices does not exceed 5.

This Class B digital device complies with Canadian ICES-003.

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Copyright © 2015 Serene Innovations, Inc. All rights reserved.