**NL
100**

User Guide

Important Safety Instructions.....	4
Set up	6
Using the NL100	7
Maintenance and Warranty	8

FCC Compliance

The FCC requires that you will be advised of certain requirements involving the use of this equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Privacy of communications may not be ensured when using this device. Changes or modifications not expressly approved in writing by Amplicom may void the user's authority to operate this equipment.

Note: This equipment has been tested and found to comply with the limits for Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures: (1) Reorient or relocate the receiving antenna. (2) Increase the separation between the equipment and receiver. (3) Connect the equipment into an outlet on a circuit different from that to which the receiver is connected. (4) Consult the dealer or an experienced radio TV technician for help.

Shielded cables must be used with this unit to ensure compliance with the Class B FCC limits.

NL 100 Neck Loop

- 1 Neck Loop
- 2 Power ON LED (Green)
- 3 Low Battery LED (Red)
- 4 Volume Control
- 5 Power ON/OFF
- 6 Microphone

Important Safety Instructions

When using your telephone equipment, basic safety precautions should always be followed to reduce the risk of fire, electric shock and injury to persons, including the following:

1. Do not use this product near water, for example, near a bath tub, wash bowl, kitchen sink or laundry tub, in a wet basement or near a swimming pool.
2. Do not use the telephone to report a gas leak in the vicinity of the leak.
3. Use only the batteries indicated in this manual. Do not dispose of batteries in a fire. They may explode. Check with local codes for possible special disposal instructions. Save these instructions for future use.

SAVE THESE INSTRUCTIONS

Intended use

This phone is suitable for use on a public telephone network. Any other use is considered unintended use. Unauthorized modification or reconstruction is not

permitted. **Do not** open the device and attempt any repair work yourself.

Power failure – What to do

Power failure can affect your phone and equipment. Only the corded phone will work if there is a power failure.


Medical equipment

Do not use the phone near emergency or intensive care medical equipment, or during an electrical storm. If you have a pacemaker check with a medical expert before using this product.

Hearing aid compatibility

This phone works with most popular hearing aids. However, given the wide range of hearing aids available, the phone may not function fully with every hearing aid.

Telephones identified with this logo have reduced noise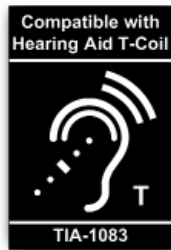

and interference when used with most T-coil equipped hearing aids and cochlear implants. The TIA-1083 Compliant Logo is a trademark of the Telecommunications Industry Association. Used under license.

Disposal

The  symbol on this product indicates that electrical and electronic apparatus and battery pack must be disposed of **separately** from domestic waste at suitable collection points provided by the public waste authorities.

Getting Started — Included Equipment

- 1 Neck Loop NL100
- 1 AAA Battery
- 1 2.5 mm to 3.5 mm Product Adapter

Set up


1. Insert the battery

- i. Insert the AAA battery into the battery compartment on the back of the NL100. Note the + and - markings and position the battery the correct way.
- ii. Close the battery compartment cover.

Important: Do not use rechargeable batteries.

2. Connect the NL100

Connect the NL100 to the headset socket of the PowerTel handset as shown in the diagram and put the neck loop over your head.

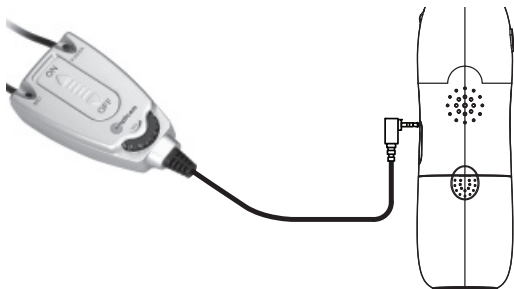Using the NL100

The NL100 uses an induction loop system which will work together with a hearing aid, when set to the T position, to help the user hear sounds more clearly.

1. Slide the ON/OFF switch to ON. The green power LED will light up.

Important: If the power LED turn red, the battery power is low. The battery should be changed as soon as possible.

2. Switch the hearing aid to T or MT mode.
3. Adjust the volume using the wheel on the NL100.
4. The NL100 is now ready for use and can be used to speak to callers with the built-in microphone.
5. Slide the ON/OFF switch to OFF at the end of the call to save battery power. The power LED will go out.

Important: The adapter cord supplied with the NL100 can be used to connect to compatible MP3 devices or mobile phones. Follow the steps above.

Maintenance and Warranty

Maintenance

- Please clean your equipment's surfaces with a soft, lint-free cloth.
- Never use cleaning agents or solvents.

Warranty

AMPLICOM equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life. The terms of the warranty do not apply where the cause of the equipment malfunction is the fault of the telephone network operator or any interposed private branch extension system. The terms of the warranty do not apply to the rechargeable battery pack or power packs used in the products. The period of warranty is 24 months from the date of purchase.

All deficiencies resulting from material or production faults which occur during the period of warranty will be eliminated free of charge. Rights to claims under the terms of warranty are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of warranty.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property.

Rights to compensation in the case of damage are excluded where there is no evidence of intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of warranty, please return to the sales outlet in which you purchased the AMPLICOM equipment together with the purchase receipt. All rights to claims under the terms of warranty in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of warranty can no longer be asserted.

This page left intentionally blank

This page left intentionally blank

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Distributed by: Amplicom USA LLC
Distribuido por: Amplicom USA LLC