

ReNew™ Dryer & Freshener User's Guide
(Hearing Aid Dryer & Battery Tester)

Thank you for your purchase of ReNew Dryer & Freshener; please read this user's guide carefully before use and keep it for future reference.

IMPORTANT SAFETY INSTRUCTIONS

WARNING – When using electric appliances, basic precautions should always be followed, including the following:

1. Read all the instructions before using the appliance.
2. To reduce the risk of injury, close supervision is necessary when an appliance is used near children.
3. Do not look directly at the two UV lights located on the front corners of the drying chamber.
4. Only use attachments recommended or sold by the manufacturer.
5. Do not use outdoors. Do not use near water.
6. To disconnect, turn all controls to the off position, then remove plug from outlet.
7. Do not unplug by pulling on cord. To unplug, grasp the plug, not the cord.
8. Unplug from outlet when not in use and before servicing or cleaning.
9. Do not operate any appliance with a damaged cord or plug, or after the appliance malfunctions or is dropped or damaged in any manner. Return appliance to the nearest authorized service facility for examination, repair, or electrical or mechanical adjustment.

