SERENE<sup>®</sup>  
innovations  
*Making Life Easier*

# SS-201 Super Bed Shaker

Operating Guide

 **Bluetooth<sup>®</sup>**

Model  
**SS-201**

Thank you for purchasing the **Super Bed Shaker, Model SS-201**. Its free App comes with a built-in alarm clock function. SS-201 provides vibrating notification on alarm clock, text messaging and other social media App's. Please read this operating guide very carefully before use and keep it for future reference. If you have any questions, please contact our Customer Care Department.

**Toll-Free Help Line:** 866.376.9271 (8AM-5PM PST, M-F)

### **Important Safety Instructions:**

In order to reduce the risk of fire, electrical shock or injury to people when using the Super Bed Shaker, Model SS-201, basic safety precautions should always be followed, including the following:

1. Read all instructions carefully before using this product.
2. Do not place unit on a metal surface. Metal surfaces can reduce wireless signal transmission.
3. This product uses rechargeable batteries.
4. To reduce risk of injury, close supervision is necessary when the appliance is used near children.

## **Important Safety Instructions (cont.):**

5. Do not allow the machine to get in contact with moving parts or exposed metal surfaces.
6. Only use attachments recommended / sold by the manufacturer.
7. Do not use product outdoors.
8. Do not use product near water.
9. To disconnect, switch all controls to **“Off”** position, then remove plug from outlet.
10. Do not unplug the unit by pulling the cord – grasp plug, not cord.
11. Unplug unit from outlet when not in use and before servicing or cleaning.
12. Do not operate the product with a damaged cord or plug; do not operate the appliance if malfunctioning occurs or is dropped or damaged in any manner. Return problem unit to the nearest authorized facility for examination; repairs may include but are not limited to electrical and mechanical adjustment.
13. Avoid using this product during electrical storms as the risk of electrical shock is increased.
14. Specifications surrounding this product are subject to change without notice.

## Important Safety Instructions (cont.):

### Disposal of SS-201 used batteries:

Please dispose of the SS-201 batteries properly, following any local regulations. **DO NOT** dispose of the unit/used batteries in the household trash. Take the equipment to a near by recycle center.

### Packaging Contents:


Your **Super Bed Shaker** is packaged with all the items shown. If anything is missing, please contact our Customer Care Department immediately\*.

Super Bed Shaker

Quick Guide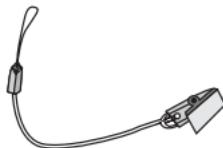


Pillow Clip

USB to Micro USB  
Charging Cable

## Feature Identification:

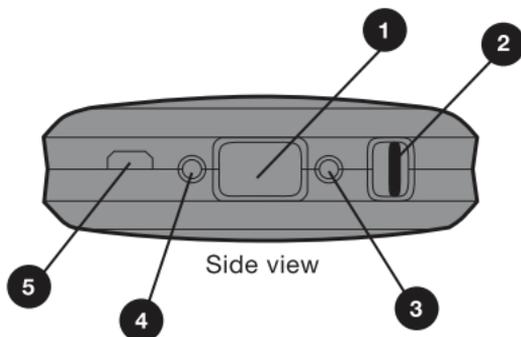1. **RESET/SNOOZE**
2. Power **ON /OFF**
3. Bluetooth® Indicator
4. Low Battery/ Charge Indicator
5. Charger Port

## Charging:

Plug the USB charging cable to an AC Adapter (not included) and connect the cable to the Super Bed Shaker Model SS-201. Allow battery to charge for approximately 3-4 hours before first use. A fully charged battery can last 10-45 days depending on frequency of use. The shaker should never be connected to the AC power adapter for more than one week.

## Charging (cont.):

### Low Battery / Charge Indicator:

- Low battery status – blinks every 3 seconds
- Charging – blinks every 1 second
- Full charge with AC adapter – solid lit

### Download App:

1. Make sure your smartphone or connecting device is Bluetooth® compatible (4.0 and above).
2. Go to Apple Store® or Google Play Store® and search for the “Super Bed Shaker” App.
3. Download the free App on your iOS or Android® device.

### Pairing Super Bed Shaker to your Smartphone:

1. Turn “**ON**” your smartphone’s Bluetooth® function and open the Super Bed Shaker App.
2. Turn “**ON**” the power to Super Bed Shaker unit.


### Pairing Super Bed Shaker to your Smartphone (cont.):

3. The App will search for the Super Bed Shaker unit.
4. Press “**Pair**” on smartphone when prompted. When successful, the Bluetooth® indicator on the Super Bed Shaker unit will blink once every 6 seconds, and the Super Bed Shaker App banner will turn Green.

### Indications of No Bluetooth Connection:


1. If the Super Bed Shaker Unit is not connected to a Bluetooth® device it will blink once every 3 seconds.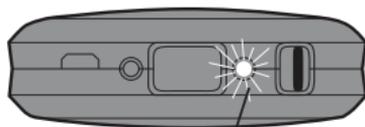

Bluetooth® Indicator

## Indications of No Bluetooth® Connection (cont.):


2. On the Super Bed Shaker App, the settings screen will say “**NOT CONNECTED**” and the RED banner will indicate no Bluetooth® connection has been established between your smartphone and the Super Bed Shaker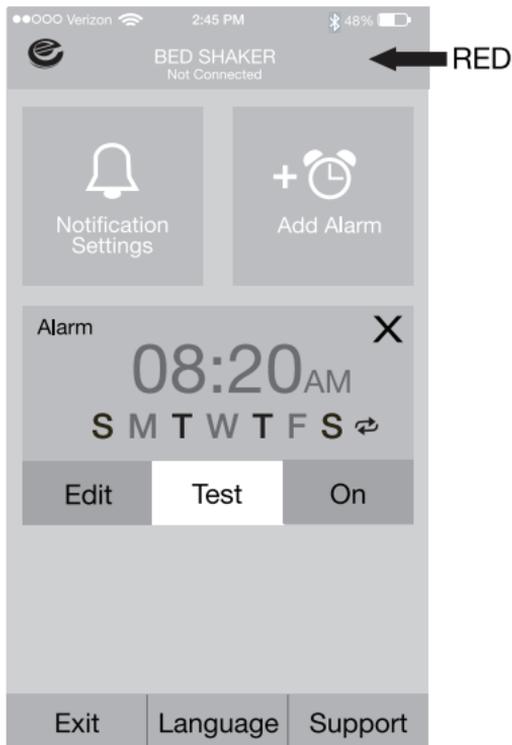

## Indications of No Bluetooth® Connection

(cont.):

3. The test screen will show that the Super Bed Shaker is not connected to the Bluetooth® device.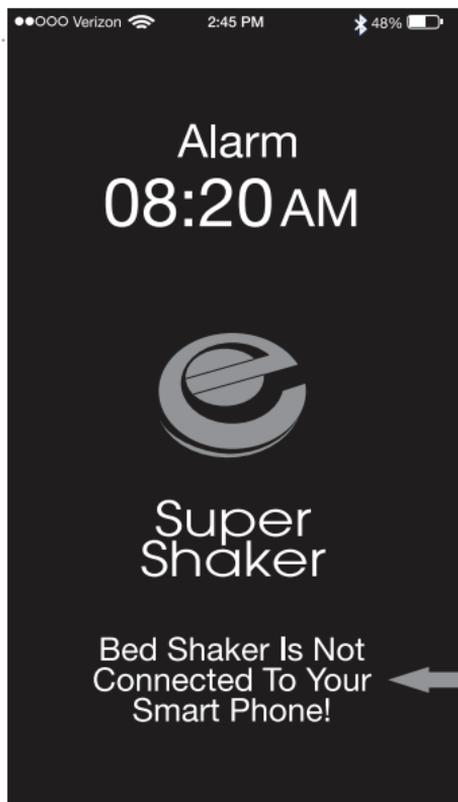


## Indications of No Bluetooth® Connection

(cont.):

4. Repeat Pairing process. Once connected, continue on App to Select Language, Alarm Times, and Notifications for Social Media.

### Language Select:

1. Tap on the screen where it says “**LANGUAGE**”.

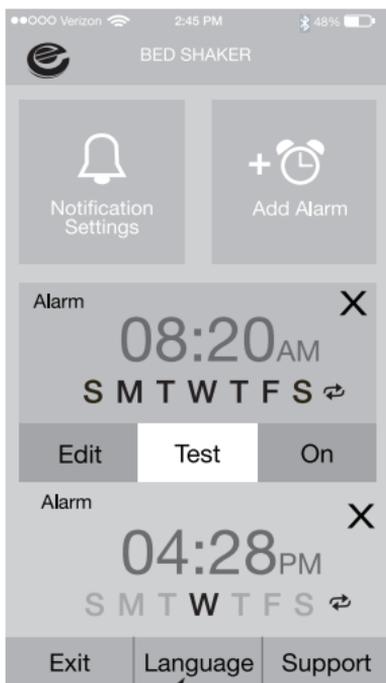
## Language Select (cont.):

2. Tap to select and highlight the language you wish to use.
3. Tap **“SAVE”** to change the language.

## Set Alarm Times:


1. Tap “**ADD ALARM**” icon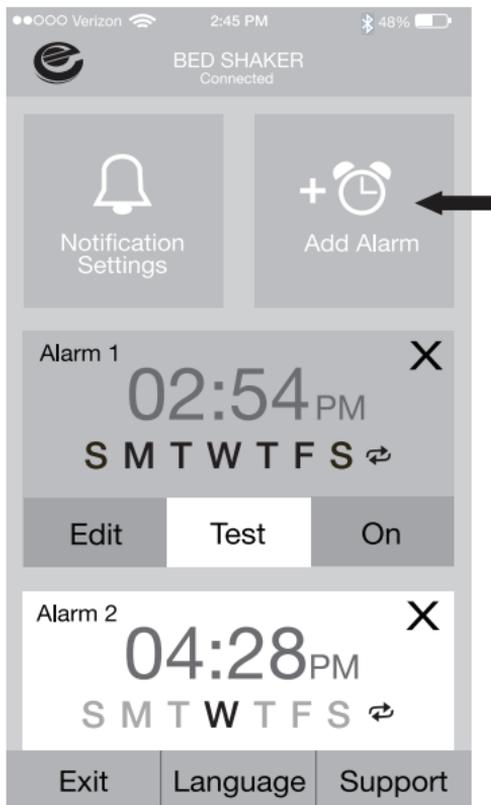

## Set Alarm Times (cont.):


2. Tap Set Alarm then tap **“SAVE”** to finish alarm setting. Display will return to main screen with updates.

**Note:** All alarm settings from the App will be saved in the Super Bed Shaker internal memory as long as its power remains on. Once downloaded, shaker can work without smart phone. Any changes to alarms, however, must be made on the App.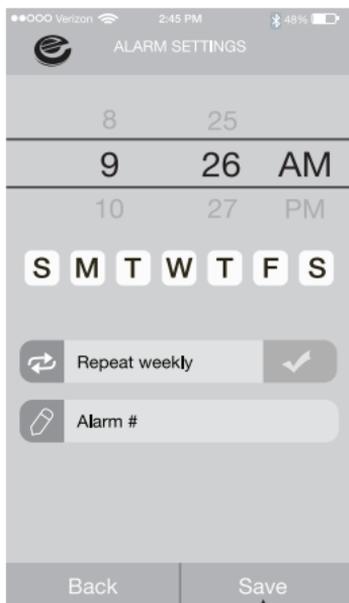

## Select Alert Notifications:

1. Tap “**NOTIFICATION SETTINGS**” to setup the alerts for incoming Calls, Text, SMS, E-mail and Social Media Apps that will set off the Super Bed Shaker.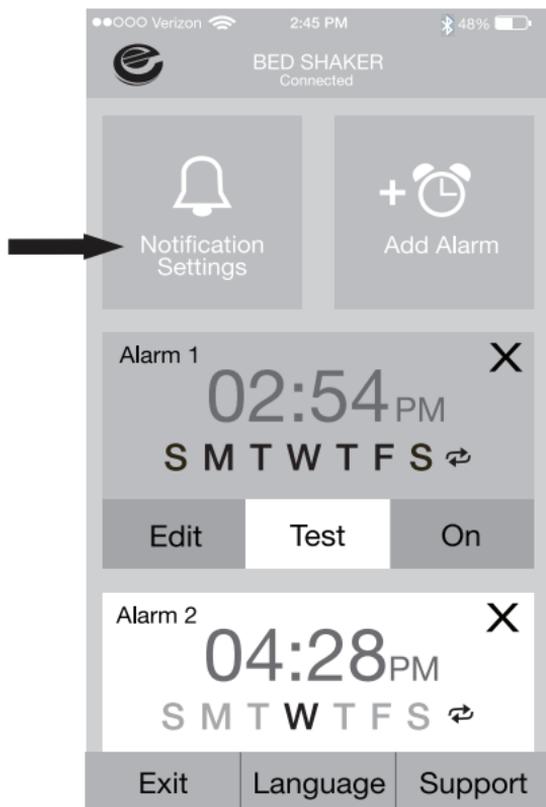

## Select Alert Notifications (cont.):

2. Tap the individual settings/Apps that you want to enable/disable.
3. Tap **“SAVE”** to save your settings.


## Super Bed Shaker Unique Vibrations:

1. For Incoming Calls – it vibrates on incoming calls. Press **“RESET/SNOOZE”** button on Super Bed Shaker or answer the call on your smartphone to end vibration.
2. For Text and Social Media Notifications - it vibrates up to 3 times.
3. For Alarm Clock Alerts – it vibrates for about 85 seconds. **“RESET/SNOOZE”** feature will stop vibration and can be repeated a maximum of 12 times for each alarm.

## Stop Vibration And Activate Snooze:


1. Press the **“RESET/SNOOZE”** button once on the Super Bed Shaker while the alarm is active to initiate a 5-minute snooze period.  
Or,
2. In the App, drag the **“RESET/SNOOZE”** button from left to right as it appears on smartphone to initiate a 5-minute snooze period.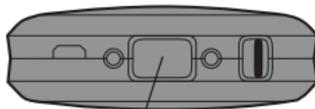

RESET/SNOOZE

## Stop Vibration and Cancel the Alarm:

1. To cancel an alarm/vibration on the Super Bed Shaker, press and hold the “**RESET/SNOOZE**” button on the Super Bed Shaker for 2-seconds.  
Or,
2. In the App, drag the “**OFF**” button from right to left as it appears on the smartphone.

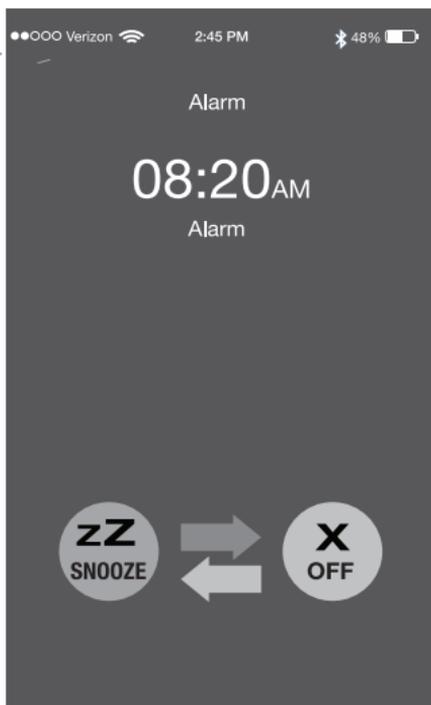**Note for Apple iPhone® Users:**

1. Once the shaker has been paired with one iPhone, each time the shaker is powered **ON** it will automatically connect to the same iPhone® in range.
2. If the shaker has been paired with multiple iPhones, each time the shaker is powered **ON** it **WILL NOT** automatically reconnect to the last synced iPhone®. Instead, the shaker will seek the Bluetooth® devices in range, and will connect to the first iPhone® that it can find, provided it has already been synced with it. The connection usually takes seconds to form.
3. The shaker can be paired with up to 5 Bluetooth® devices, after which it will start to overwrite the old devices.

**Note for Android® Users:**

1. If the shaker was paired with an Android® device, each time the shaker is powered **ON** it **WILL NOT** automatically connect to the last synced Android® device. This is because Android® phones vary in their models and operating systems.

**Note for Android® Users (cont.):**

2. Different Android® models may selectively close non-active Apps to save battery power. To ensure successful connection, activate the shaker's App before you power **ON** the shaker to reconnect. Please test your Android® devices for automatic reconnections and decide if this extra step of Super Bed Shaker App reactivation is necessary for your device.
3. The shaker can be paired with up to 5 Bluetooth® devices, after which it will start overwrite the old devices.

**Cleaning And Maintenance:**

1. The surface of the device can be cleaned with a soft, dry cloth.
2. Do not use any sprays near the device. Do not use any solvents, chemicals, or cleaning solutions containing alcohol or ammonia or abrasives.
3. Do not allow liquids to spill into any openings on the device.

**Technical Specifications:**

Bluetooth.....	Version 4.0 or later
Syncing.....	iOS 8 and up Android 4.4 or later
Display.....	None
Transmit Power.....	Class 2
Paired Devices.....	Up to 5 (max.)
Effective Range.....	Approx. 10m (33 feet)
Battery Installed.....	3.7V rechargeable
Usage Time.....	Up to 45 days based on usage
Battery Charge Time.....	Approx. 3- 4 hours
Charging cable.....	USB type to micro USB
Charging Voltage.....	5VDC 500mA 5VDC USB AC adapter (not included)
Operating temperature...	10° to 45° C
Dimensions.....	Approx. 78 x 90 x 26mm
Weight.....	Approx. 4.5 oz. (with battery)

Specifications are subject to change without notice

## Warranty / Service Information:

Your **Super Bed Shaker (SS-201)** comes with a one-year limited warranty. For the warranty period we will guarantee any defects involving materials or production relating to the **Super Bed Shaker (SS-201)**. In the event that the **Super Bed Shaker (SS-201)** fails to function properly, and if it is during the one-year warranty period, please return the unit (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to Serene Innovations.

Serene Innovations will either repair or replace the unit (with a refurbished unit or a unit of equal condition) and return it to you (using UPS/USPS ground shipping) at no cost to you (there will be a nominal additional shipping charge if it is shipped outside of the 48 Continental states).

This warranty does not apply to any product that has been accidentally damaged due to: abuse, misuse, negligence, use on an electrical frequency or voltage other than the one marked on the product and/or described in this manual. Defects or errors caused by unauthorized alterations, repairs and/or tampering are also not covered by this warranty.

Please dispose of used batteries properly, following any local regulations. Do not incinerate.

### **Incidental of Consequential Damages:**

Neither Serene Innovations, Inc. nor the retailer, dealer or selling distributor has the responsibility for any incidental or consequential damages, including, without limitation, commercial loss of profit or for any incidental expenses, loss of time or inconvenience. Some states do not allow exclusion or limitation of incidental or consequential damage, so the above limitation or exclusion may not apply to you.

This warranty does not apply to any product that has been accidentally damaged, abused, misused or neglected, used on electrical frequency or voltage other than marked on product and/or described in the manual. Defects or errors caused by unauthorized alterations, repairs and/or tampering, are also not covered by this warranty. This Warranty grants you specific legal rights and you may also have other legal rights that vary from state to state. If a defect covered by this warranty should occur, promptly contact a Customer Care Representative by phone, or log onto [www.sereneinnovations.com](http://www.sereneinnovations.com) to obtain a Return Merchandise Authorization Number (RMA) and shipping instructions, before shipping the product to Serene Innovations. Any shipment without an RMA number will not be accepted and will be returned to you at your expense.

**Incidental of Consequential Damages (cont.):**

Any authorized returned product must be accompanied by a proof of purchase and a brief explanation of the problem.

For out-of-warranty repair and service, please contact our Customer Care Department for a price quote and instructions.

**FCC And IC Information:**

This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. This equipment has been tested and found to comply with the limits of a Class B digital device, pursuant to Part 15 of FCC rules. These limits are designed to provide reasonable protection against harmful interference in residential installation.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television

**FCC And IC Information** (cont.):

reception, which can be determined by turning the equipment off and on, you may try one the following methods to correct the interference:

1. Where it can be done safely, re-orient the receiving television or radio antenna.
2. To the extent possible, relocate the television, radio, or other receiver, with respect to the cordless phone.
3. Connect the cordless phone AC adapter into a different electrical outlet.
4. Consult the dealer or an experienced radio/TV technician for help. Modification: Any modification not expressly approved by the manufacturer of this device could void the user's authority to operate the device.

**This Class B digital device complies with Canadian ICES-003, RSS 201 Issue 8.**

iPod and iPad are trademarks of Apple Inc.

Bluetooth is a trademark of Bluetooth SIG, Inc.

Android is a product of Google

Google Play and the Google Play Logo are trademarks of Google Inc.

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Copyright © 2017 Serene Innovations, Inc. All rights reserved.
iPod® and iPad® are trademarks of Apple Inc.
Bluetooth® is a trademark of Bluetooth® SIG, Inc.