



KA1000NR



Nursery Room Call Transmitter



Instruction Booklet

01.E.2014-05-01

KA1000NR - Nursery Room Call Transmitter

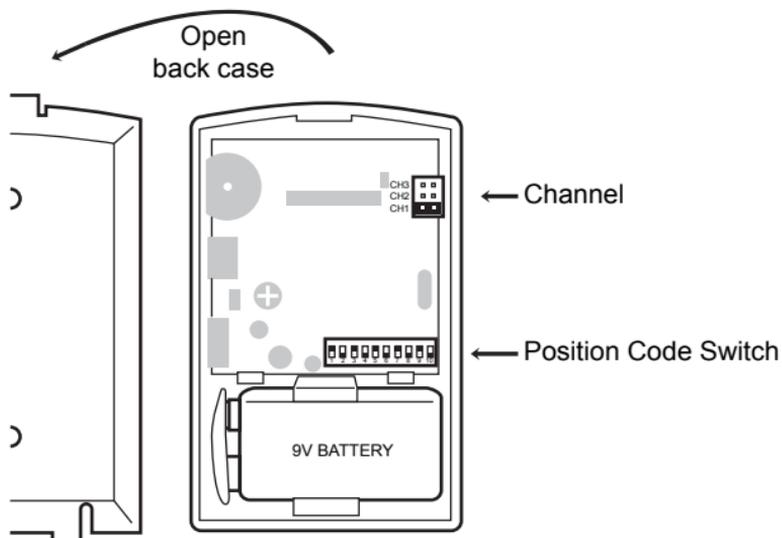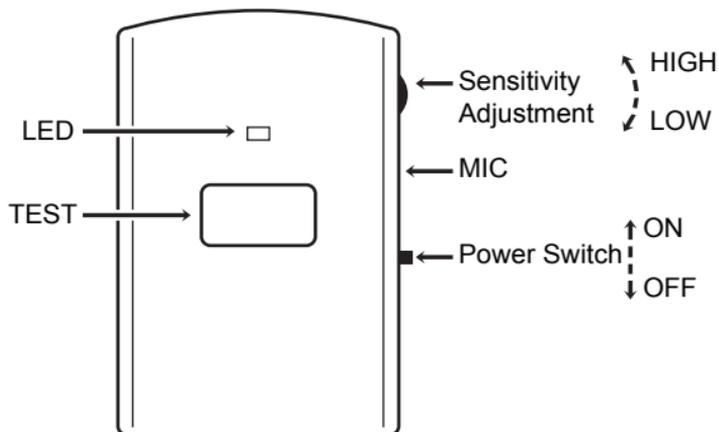

Getting Started - Transmitter pairing and testing

- Remove back casing of the unit and verify 10 position code switch matches with the corresponding receivers. (KA1000 alarm clock receiver or KA1000RR remote receiver)
- Install 9 volt (1604 type) battery and turn on the unit. (Slide the POWER switch to the ON position).
- Press the TEST button of the unit for a minimum of 3 seconds. The LED indicator will turn ORANGE when the unit is transmitting for 3 seconds or more.
- Ensure the corresponding receiving unit activates indicating a received signal from the KA1000NR transmitter.

Placement

- Place the KA1000NR with MIC (microphone) as close to the baby as possible and not more than 5 feet away.
- Place the KA1000NR with the MIC (microphone) pointing towards the source of the sound, audio alarm, or door chime being monitored.

Sensitivity adjustment

- Activate the device being monitored. (e.g. chime/alarm) the LED will illuminate GREEN when the MIC picks up sound.
- If the LED does not come on then move the sound sensitivity adjustment to a "HIGH" sensitivity level and test again.
- If the MIC is triggered for 3 seconds or more the LED of the unit will illuminate ORANGE.

Battery replacement recommended every 6 months or when a low battery condition exists

- The KA1000NR has a low battery detect feature and the LED will turn RED during transmission.
- The display of the KA1000 and the RED LED on the KA1000RR will also flash when receiving a signal from a transmitter with a low battery condition.

LOCATION: **Krown Manufacturing, Inc.**
3408 Indale Road, Fort Worth, TX 76116, United States of America

OFFICE: +1 (817) 738-2485 (voice/TTY)
VP: +1 (682) 200-1331
FAX: +1 (817) 738-1970
EMAIL: techsupport@krownmfg.com