

NOTES:

When all the desiccant beads are green, it is time to reactivate. To do this, place desiccant, screen side up, in microwave & heat for 30 seconds on high. **COOL FOR 20 MINUTES** . Most beads will have a yellowish tint which means it is ready to use. If not heat for another 30 seconds.

In a conventional oven, remove desiccant from jar and bake, screen side up, for 1 hour at 200°F (93°C), or until beads are yellow. Allow desiccant to cool before handling.

**DO NOT Place Hearing Aid
in Microwave or Oven**

DIRECTIONS:

1. Before retiring for the night, clean your hearing aid (and ear molds, if you have them) with Audiowipes™ disinfectant wipes.
2. Open battery doors.
3. Place the hearing aid on the screen side of desiccant.
4. Replace the lid and close tightly.

(OVER)