



Presentation

Otofloss is indicated for cleaning and drying acoustic tubes in hearing systems, for which brushes are not appropriate. To this extent, they prevent the loss of sound quality while preserving the effectiveness of hearing instruments.

Otofloss are wadded yarns

Properties

The otofloss meet the requirement of Class I Medical Devices, according to the Regulation 2017/745/EU related to Medical Devices, while respecting environmental and safety for health.

As manufacturer of Medical Devices, MG Development makes sure of the origin of the raw materials of all its products. And so, all the materials and the coloring agents are complying with the directives and the existing standards (**REACH** for the restriction of chemical substances).

Using advices

Remove the sound tube from the hearing instrument, then from the dome/tip.

Insert the rigid end of the otofloss into the part of the sound tube that is usually attached to the hearing instrument.

Push the otofloss through the tube until it comes out on the other side.

Pull the otofloss through the entire tube.

Precautionary statements

Single use only.

Keep out of reach of children.

Product composition

Nylon