



Sonic Alert

America's Leader in Visual Alerting Systems. Model SBT425SS-v3

Sonic Boom Dual Alarm Clock with Telephone Signaler & Bed Shaker and 5-Level Dimmer



OPERATION INSTRUCTIONS

IMPORTANT

- Please read these instructions carefully before use and retain for future reference. See operating instructions on back.
- Use only the supplied or recommended Sonic Alert adaptor to connect the unit.

Warning:

To prevent fire or shock hazard do not expose this appliance to rain or moisture.

Before connecting to mains, make sure that the requirements specified on the apparatus corresponds to the available power supply in that country i.e.

AC100-240V~, 50/60 Hz.

- The ventilation should not be impeded by covering the ventilation openings with items such as, newspaper, table cloth, curtains, etc;
- No naked flame sources, such as lighted candles should be placed on the apparatus.
- It shall not be exposed to dripping or splashing and that no objects filled with liquids, such as vases, shall be placed on the apparatus.
- Do not place the product in closed bookcases or racks without proper ventilation.
- Where the mains/ power plug is used as the disconnect device, the disconnect device shall remain readily operable.

GENERAL DATA

UL listed Power Adaptor: AC100-240V~, 50/60Hz


Power Consumption: Adaptor Output: DC9V \equiv , 650mA

Test Mode:

- 1) Press and hold the Snooze and Time buttons for 3 seconds to enter the Test Mode. The display will show "7ES7" to indicate it is in TEST Mode.
- 2) If within 1 minute no button is pressed, then it will exit the Test Mode and return back to the Time Mode.
- 3) Press any button within 1 minute (except the Snooze & Time buttons, which has no effect), it will exit the Test Mode immediately.
- 4) While in Test Mode, sliding the Function Switch will demonstrate the different functions of the clock:
 - a) Function Switch at OFF, it will have a continuous "Vee" sound to indicate it is in the Test Mode.
 - b) Function Switch at BUZZ, it will demonstrate the buzzer sound.
 - c) Function Switch at VIB, it will demonstrate the vibrator.
 - d) Function Switch at VIB & BUZZ, it will demonstrate the buzzer sound & the vibrator.
- 5) After finish with the Test Mode, press any button (except the Snooze & Time buttons, which has no effect) it will exit the Test Mode immediately.

INSTRUCTIONS

Model SBT425SS-v3

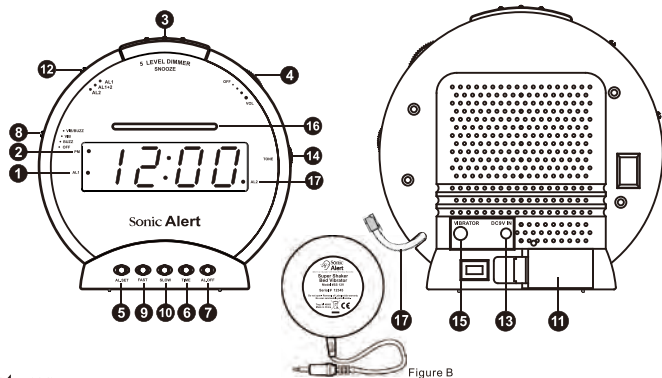

Features

- | | | |
|---|--|----------------------------|
| 1) Alarm 1 Indicator | 6) Time Set Button | 13) DC Adaptor Input |
| 2) PM Indicator | 7) Alarm Off Button | 14) Tone Control Wheel |
| 3) Snooze button / Snooze Time Adjustment/ Dimmer Control | 8) Alarm Function Switch | 15) Vibrator Output |
| 4) Volume Wheel | 9) Fast Min./ Hour Button / 12 or 24 Hour Selector | 16) Flashing Light Display |
| 5) Alarm Set / Alarm Duration Adjustment | 10) Slow Min./ Hour Button | 17) Alarm 2 Indicator |
| | 11) Battery Compartment | |
| | 12) Alarm Mode Switch (AL1, AL2, AL1+AL2) | |

Setting the Correct Time –

- 1) Press and hold the “TIME” Button (6) and at the same time press the “FAST” Button (9) or the “SLOW” button (10) to advance clock display until you have the desired time.
- 2) The lighted dot (2) located in upper left corner of the clock display indicates the time is PM. For AM time, this dot will not light up.

Setting the Alarm Time –

Setting Alarm 1

- 1) Slide the “ALARM MODE SWITCH” (12) to AL1 position, the alarm indicator “AL1” (1) will light up.
- 2) Press and hold the “ALARM SET” (5) button and at the same time press the “FAST” Button (9) or the “SLOW” button (10) to advance clock display until you reach desired alarm setting.
- 3) Make sure alarm time is correctly set for AM or PM (1).

Setting Alarm 2

- 4) Once alarm AL1 has been set, slide “ALARM MODE SWITCH” (12) to AL2 position, the alarm indicator “AL2” (17) will light up.
- 5) Press and hold the “ALARM SET” (5) button and at the same time press the “FAST” Button (9) or the “SLOW” button (10) to advance clock display until you reach desired alarm setting.

Alarm –

Enable Alarms:

- 1) To enable Alarm 1, slide the “ALARM MODE SWITCH” (12) to AL1 position, the alarm indicator “AL1” (1) will light up.
- 2) To enable Alarm 2, slide the “ALARM MODE SWITCH” (12) to AL2 position, the alarm indicator “AL2” (17) will light up.
- 3) To enable both alarms, slide the “ALARM SELECTOR” (1) to AL2 position, the alarm indicators “AL1” (1) and “AL2” (17) will light up.

To Select Wake up mode –

To select the wake up mode, slide the “Alarm Function Switch” (8) to the desired position:-

- a) “BUZZ” for sound
- b) “VIB” for bed shaker
- c) “VIB/ BUZZ” for both sound and bed shaker

NOTE: once the wake up mode is chosen, this will apply to both alarms.

Alarm Patterns –

When the alarm is activated, alarm pattern of the buzzer/ flashing display will be out of sync with the vibrator. This means that when the buzzer / flashing display is ON, the vibrator will be OFF and while the buzzer/ flashing display is OFF, the vibrator will be ON.

ALARM 1 - When Alarm 1 is activate, it will have 3 beeps for buzzer, then the vibrator. This pattern is repeated until the alarm is stopped.

ALARM 2 - When Alarm 2 is activate, it will have 4 beeps for buzzer, then the vibrator. This pattern is repeated until the alarm is stopped.

Adjusting The Alarm Duration Time –

The default alarm duration time setting is 30 minutes; you may adjust the alarm duration between 1 minute and 59 minutes as desired. To adjust the alarm duration:

- * Press and hold the **AL. SET button (5)** for six seconds. The clock display will show the default alarm duration as '30'. Release the AL set button
- * Press the “MIN” button (10) to adjust the alarm duration to the desired setting, from 1 minute to 59 minutes.
- * When the display is showing the desired alarm duration time, it will return to the normal clock mode after 3 seconds. The new alarm duration time setting becomes the new default and remains in the memory until you change it again.

Snooze Operation –

1. When the alarm turns on in the morning you may press the **SNOOZE Button (8)** to stop the alarm temporarily. The alarm will shut off for 9 minutes or for the predetermined time you select from 1-30 minutes, and then turn on again. The snooze operation may be repeated indefinitely until the alarm is turned off.
2. The SNOOZE function of Alarm 1 and Alarm 2 is independent of each other.

Adjusting The Snooze Time –

- The default snooze time setting is 9 minutes but you may adjust the snooze time between 1 minute and 30 minutes as desired. To adjust the snooze time:
- * Press and hold the **SNOOZE Button (8)** for four seconds. The clock display will show the default snooze time '9'. Release the snooze button.
- * Press the “SLOW” Button (7) to adjust the snooze time to the desired setting, from 1 minute to 30 minutes.
- * When the display is showing the desired snooze time, it will return to the normal clock mode after 3 seconds. The new snooze time setting becomes the new default and remains in the memory until you change it again.

Shutting Off the Alarm –

Alarm can be shut off temporary for the day or permanently shut off.

- 1) When alarm goes off, press the “ALARM OFF” (7) button to turn off for the day and come back on at the same time the next day, if alarm settings have not changed.
- 2) To shut the alarm off permanently, slide the “Alarm Function Switch” (8) down to the OFF position. The alarm will not come on until the “Alarm Function Switch” (8) is set to “VIB/BUZZ”, “VIB”, or “BUZZ” position.

NOTE: The Alarm Off function of Alarm 1 and Alarm 2 is independent of each other.

Dimmer Operation –

There is a 5-level brightness control for the LED display. Press the SNOOZE button once to change the display brightness. The brightness will cycle from 5-4-3-2-1-2-3-4-5 (from bright to dim then from dim to bright).

NOTE: The dimmer function will not work, when the clock is in snooze mode.

Volume Control –

- 1) To adjust the volume, rotate **VOLUME wheel (4)** located on the lower right corner of the clock.

Tone Control –

- 1) To adjust the TONE, turn the **TONE wheel (14)** located on the top right corner of the clock.

Bed Shaker / Vibrator –

- 1) Attach the bed shaker jack to the back of clock at “VIBRATOR INPUT” (15).
- 2) To turn on vibrator slide “Alarm Function Switch” (8) to “VIB/BUZZ” or “VIB” position.
- 3) Place vibrator between mattress and box spring.

Battery Backup –

To use the battery backup, you can install a 9-volt alkaline battery in the back of the clock located under the **Battery Compartment Cover (11)**. The battery needs to be purchased separately. **Note the LED display will not show during power failure but the clock will still be running and keeping proper time until the power comes back.**

During a power failure, the alkaline battery can only keep the clock IC running for up to 30 days. **(During a Power failure, the LED display will not show the time.)**

NOTE: For the best results it is recommended that the battery be changed every 6 months. If the clock is not used for a while, it is recommended to remove the battery to avoid damage to the clock from battery leakage..

DC 9V Power Input –

Attach the “DC POWER INPUT” (13) to the back of DC9V IN next to “VIBRATOR INPUT” (15); it is the smaller input of the two.

NOTE: IMPORTANT: do not confuse “DC POWER INPUT” (13) with the “VIBRATOR INPUT” (15)

OPERATION OF TELEPHONE SIGNALER–

Your telephone signaler alerts you that your telephone is ringing by flashing the light above the clock display it will also set off the bed shaker and buzzer alarm sound automatically **when the phone rings.**


Telephone Signaler Installation –

- 1) Plug the modular phone plug from the back of the clock into a standard modular phone jack. (Use Jack type USOC RJ11C) the seven-foot cord is long enough to accommodate a variety of phone locations. However if the phone outlet is further away from the electrical outlet and you need a longer telephone line extension lines are available at Radio Shack or hardware stores.
- 2) Plug the clocks power adaptor into any standard electrical outlet.
- 3) Adjust the volume (4) with the volume wheel on the right hand side.
- 4) Adjust the Tone (14) with the wheel on the right hand side just below the volume.
- 5) Set function switch (8) to desired position for BUZZ or VIB or both.
- 6) Use a phone 2 to 1 adaptor (splitter) if you need to use the phone outlet with a telephone and the SBT425 signaler.

Your SBT425ss complies with Part 68 of FCC Rules. You must, upon request, provide the FCC registration number and the REN to your local phone company. Both numbers are shown on the TR75's rating label.

Modifying or tampering with your TR75's internal components can cause a malfunction and invalidate the TR75's warranty and void your FCC authorization to operate it. If the TR75 is not operating as it should, return it to Sonic Alert.

If the problem is harming the phone lines, the phone company might ask that you disconnect the TR75 until the problem is resolved.

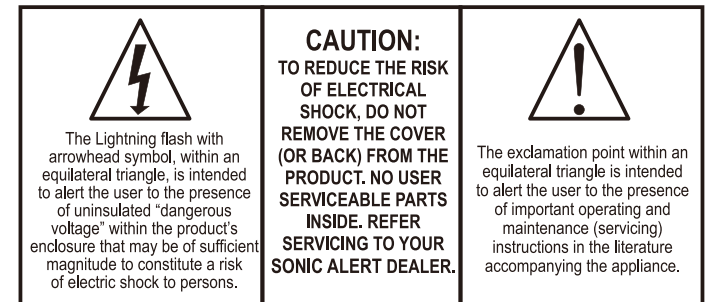

WARNING: TO PREVENT FIRE OR SHOCK HAZARD, DO NOT EXPOSE THIS APPLIANCE TO RAIN OR MOISTURE.

- Attention should be drawn to the environmental aspects of battery disposal.



- this indicating separate collection for electrical and electronic equipment.

Normal function may be disturbed by strong electromagnetic interference. If this is the case, normal function can be resumed by simply resetting the unit following the procedure in the instructions manual. If function does not resume, try moving the unit to another location.

SONIC ALERT'S 1 YEAR LIMITED WARRANTY

The Sonic Boom Alarm Clock Model SBT425ss is warranted against manufacturing defects in materials and workmanship for one (1) year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the SBT425ss without charge for parts and labor. Simply contact our customer service department to obtain an RA (return authorization number) and instructions on how to process your product for return. Call 1-888-864-2446 or e-mail to RMA@sonicalert.info. You'll need a copy of your receipt to process the return authorization.

Sonic Alert Inc., Troy MI 48083, USA.

Sonic Alert

Made in China

Sonic Alert Troy, MI 48083

www.SonicAlert.com 1-888-864-2446 V/TTY

