



## Installation and Operating Instructions

# ***Model TEL-MC***

*Telephone Transmitter 418MHz*

### **Installation:**

1. The telephone transmitter will work with standard telephone lines and with video phones.
2. **To install the TEL-MC telephone transmitter on a conventional telephone line**, plug the telephone transmitter into any telephone outlet. Use the supplied dual adapter if you need to connect a telephone AND the transmitter to the same outlet.
3. **To install the TEL-MC telephone transmitter on a videophone**, plug the telephone transmitter into the telephone outlet on the videophone. Use the supplied dual adapter if you need to connect a telephone AND the transmitter to the videophone.
4. If plugged into a two line telephone outlet, the telephone transmitter will respond to line one only.
5. The TEL-MC transmitter may not work with digital phone systems. Please contact your telephone service provider as filters may be necessary to filter out the modem tones associated with digital (DSL) or VOIP type services.

### **Operation:**

1. Whenever the telephone (or videophone) rings, the Red light on the telephone transmitter will light up and the transmitter will send a signal to your Silent Call Medallion Series Receiver causing the receiver to activate.
2. Transmission range with a Medallion Series receiver is up to 500 ft.
3. This unit is powered by two AA alkaline batteries (included) which should last a year or more depending on use.
4. When the Yellow LED (low battery indicator light) on the face of the transmitter comes on it's time to replace the batteries.

### **Transmitter Settings:**

The Silent Call Medallion system is digitally encoded. All Silent Call Medallion Series receivers are capable of receiving one million different codes and the transmitters are preset at the factory to send one of the million different codes. Please refer to your receiver manual for instruction on teaching the transmitter code to the receiver.

### **Replacement Batteries:**

Use two AA 1.5V Alkaline Batteries. AA Batteries may be purchased from local retail stores. Transmitter is shipped with 2 Rayovac Alkaline Batteries Type LR6. Batteries may be purchased Online at [www.silentcall.com](http://www.silentcall.com).

### **Technical Support**

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at [support@silentcall.com](mailto:support@silentcall.com)

### **Limited Warranty**

Your transmitter is warranted to be free of defects in material and workmanship for five years from the date of initial purchase. During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. is warranty is void if the defect is caused by customer abuse or neglect.



Return for Serving To:

### ***Silent Call Communications***

5095 Williams Lake Road, Waterford Michigan 48329

(800) 572-5227 v/tty (248) 673-7360 fax

Website: [www.silentcall.com](http://www.silentcall.com) Email: [silentcall@silentcall.com](mailto:silentcall@silentcall.com)