

# Installation and operations manual

## *Omni Page Receiver*

Model GV1097-1

### **General**

The Silent Call Omni Page™ receiver is a body worn portable alerting device with vibrating and visual alerts. It is used in conjunction with any Silent Call transmitter such as the telephone, doorbell, smoke detector and/or sound monitor. When the Omni Page receiver detects a signal from a Silent Call transmitter, it activates its vibrator. There is no visual indication on the Omni Page receiver.

### **Operation**

Switch the Omni Page receiver on by moving the power switch to the ON position. Each time it receives a signal from a Silent Call transmitter with a matching code, the Omni Page receiver will vibrate. The Omni Page receiver battery may be tested by moving the power switch momentarily to the TEST position.

The Omni Page receiver is supplied with a rechargeable Nickel Metal Hydride 9V battery. In normal operation, the unit will function for approximately two to three days per charge. However, for the most reliable operation, recharge the battery each night when you go to bed. Also, do not leave the unit on when you are not using it as the battery can be damaged by running it all the way down. Not all "9V" rechargeable batteries are the same, and we recommend you purchase replacement batteries from us. Although it is not recommended, the Omni Page receiver may be used with a standard alkaline battery. However, if you use an alkaline battery **YOU MUST NEVER ATTEMPT TO RECHARGE THE BATTERY**. Doing so will cause the battery to burst and destroy the Omni Page circuitry, voiding the warranty.

#### **"Code" (address) switches**

Each Silent Call device contains a five position "code" switch that defines the address of the device. Each receiver and each transmitter must be set to the same code. All Silent Call devices leave the factory set to a default code of 1=off, 2=off, 3=off, 4=on, 5=on. The ON direction is marked on the switch with an arrow. Ordinarily, this code will not need to be changed unless your system is within 100-200 feet of another Silent Call system. In that case, simply set all receivers and transmitters to a different code.

### **Technical Support**

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at [support@silentcall.com](mailto:support@silentcall.com)

### **Limited Warranty**

Your Omni Page receiver is warranted to be free of defects in material and workmanship for five years from the date of initial purchase (not including the battery). During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. To obtain warranty service, contact Silent Call to obtain a return materials authorization. This warranty is void if the defect is caused by customer abuse or neglect.