


# Quick Start Instructions

UNISAR®

Thank you for purchasing the Unisar® TV Listener® J3. Charge the headset for 16 hours prior to first use. For more detailed instructions, please refer to the Instruction Manual.


## A Transmitter

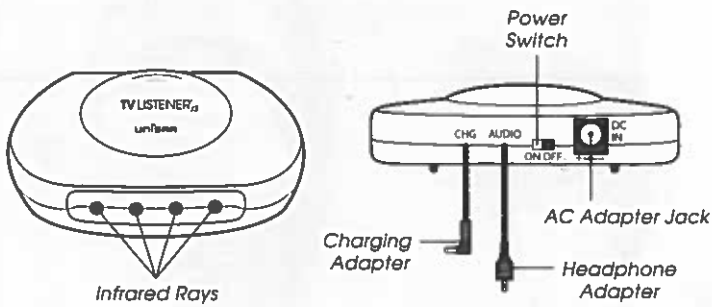

## Setting Up the System

Connect the transmitter to your television or audio device. Use the adapter appropriate for the type of audio output your device has.


- Two audio output jacks (right and left) - Use the RCA 3.5mm stereo adapter (D).
- Small headphone jack - Plug the transmitter directly into your equipment.

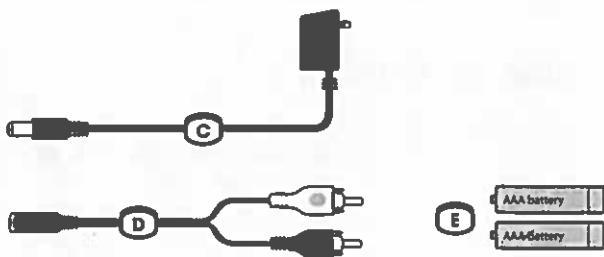

Plug the AC adapter (C) into the jack marked "DC IN" on the back of the transmitter and plug the adapter into an electrical outlet.


The headset (B) requires two rechargeable AAA batteries (E) (included). To access the battery compartments, gently pull the ear pads off the headset and insert one rechargeable AAA battery in each side. To replace the ear pads, snap them

## B Headset

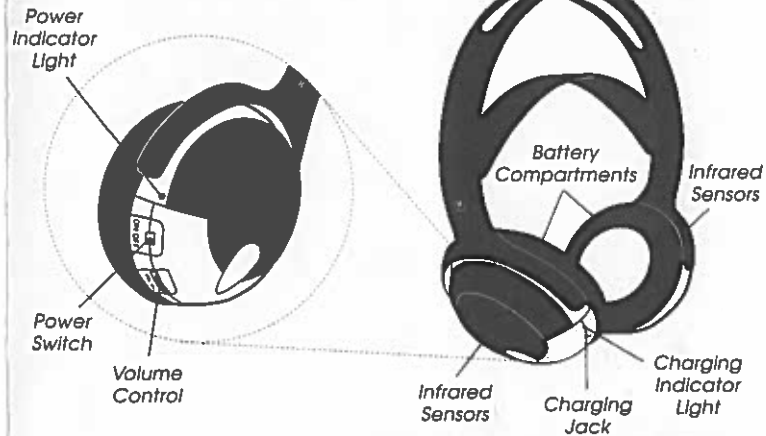

back on by lining up the prongs with the corresponding openings on the headset. **Charge for 24 hours prior to first use making sure the headset is turned off and the transmitter is plugged in to an electrical outlet and turned on.**

Plug the charging adapter from the back of the transmitter into the jack marked "CHG" on the right earpiece of the headset. The green charging indicator light on the headset will illuminate. Do **NOT** charge for more than 24 consecutive hours as it will shorten the life of the batteries.

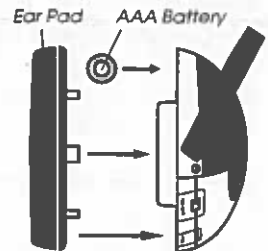

## Operating the System

Place the transmitter at or close to your eye level when you are sitting. There should be nothing obstructing the transmission of the infrared light rays from the transmitter to the receiver in your headset. The transmitter must be within 33 feet (10 meters) of the headset for the system to work properly.

Turn the transmitter on using the power switch on the back. Turn the headset on using the power switch on the right earpiece.

To prevent possible damage to your ears, be sure to lower the volume on the headset before putting it on. Put the headset on and adjust the volume to a comfortable listening level using the volume control dial.


Turn the transmitter and headset off after each use except when charging the headset when the headset should be turned off and the transmitter should be on.

For more detailed instructions, please refer to the Instruction Manual.


# Battery Compartments

UNISAR

To access the battery compartments, gently pull off the ear pad on each side of the headset where indicated by the "PULL" arrow. Be sure to insert a battery in **BOTH** sides of the headset.

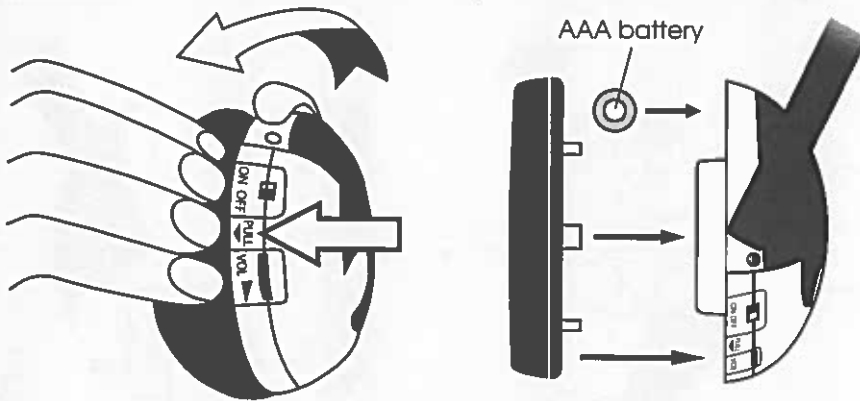

To replace the ear pads, line up the orange dots on the battery compartment with the orange dots on the inside of the ear pad. Then gently snap them back together.

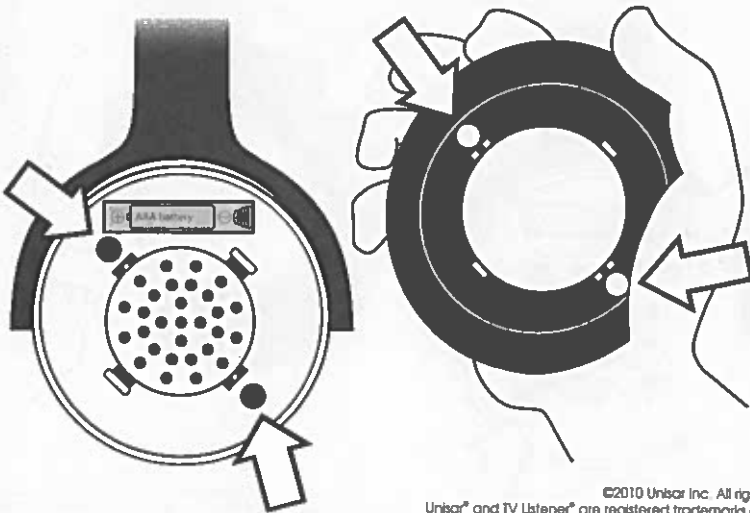

©2010 Unisar Inc. All rights reserved.  
Unisar® and TV Listener® are registered trademarks of Unisar Inc.