

OTC

OtoClip™

Prevent the loss of expensive hearing instruments

- BTE OtoClip Monaural
- BTE OtoClip II Binaural
- BTE OtoClip II with Bolo
- BTE OtoClip for Eyeglasses

Westone

Made in Mexico




How to install the OtoClip

- Disconnect the earpiece tubing from the earhook of the hearing instrument.
- Slip the earhook through the loop of the OtoClip.
- Reattach the earhook.

Repeat the above process if you have hearing instruments for both ears.

For eyeglasses, connect one end of the OtoClip to the hearing instrument, and the other end over the eyeglass temple.

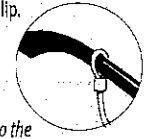

PN 91668 Rev 12